

Proper care extends the life of your aprons

To ensure the longevity of your new garment, please follow these simple guidelines:

- Radiograph your garment upon receipt and regularly after that date to ensure no damage has occurred to the protective core.
- Hang your apron properly either by the shoulders or straps.
- It is not recommended to fold, crease, or sit in your protective garment.
- Hook-and-loop fasteners should be matched together so they do not snag or tear the fabric.
- Do not expose the garment to extreme temperatures, submerge in liquids, or place in dryers.
- Do not lean or rub against sharp corners/edges or store sharp objects in the pockets.



Cleaning

Clean your protective wearables in accordance with your facility's infection control protocols.

Remove stains immediately when they occur. If this is not possible, clean with warm water and a non-bleach dishwashing soap. Do not use harsh abrasive cleaners. Some upholstery cleaners that remove protein stains may be used to clean blood.

Warranty

AliMed guarantees its aprons against defects in materials and workmanship for two years from date of factory shipment.

AliTrack

AliTrack is a web-based inspection and inventory asset management system. Visit [AliMed.com/alitrack](https://www.ali-med.com/alitrack) for more information.



Read more on the blog!

*Extending the Life of Your
Radiation Protection Aprons*

[Learn more](#)

Apron Care Products

■ Brush away tough stains

Scrubbles Apron Cleaner

Enzyme-active formula cleans barium, blood, and other common radiology-related stains from protective aprons. Simply spray it on, let it soak, and brush.

32-oz. spray bottle with brush



#2970015048 APRON CLEANER EACH
65 California residents, see below.

■ Alcohol-free

Protex® Ultra Wipes EPA-approved, alcohol-free wipes effectively kill a wide range of bacteria, viruses, and fungi, including MRSA, HIV, Staph, and many others. Useful in disinfecting hard, nonporous surfaces as well as sensitive equipment. Soft and nonabrasive with a reduced kill time of only 4 minutes.



#2970009303 ULTRA WIPES 100/CANISTER
65 California residents, see below.

■ No-slip hangers

Heavy-Duty Apron Hangers

are ideal for use with valet racks or in closets to keep aprons off floor. Lifted sides prevent garment slipping.

3/8"-diameter welded steel rod
• Chrome-plated



#2970014672 HEAVY-DUTY APRON HANGER 3/PACK
65 California residents, see below.

California Proposition 65 Statement

65 **WARNING:** This product can expose you to chemicals including Di(2-Ethylhexyl) Phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to <https://www.p65warnings.ca.gov>.