














Aspen. Infant & Pediatric Collars

A Proper Fit as Easy as "1 - 2 - 3"

- 1. Chin-Dots hidden?** The chin must be flush with the front of the chin piece plastic.
- 2. Velcro® past mark?** Back panel plastic must overlap the sides of the front panel.
- 3. FlexTabs™ bent?** The FlexTabs™ on the front and back panels should be flexed.

Sizing Guidelines (Note: New Label Colors)

		1 - 18 months	
		11 - 29 lbs. weight 5 - 13 Kg.	21 - 33 in. length 53 - 84 cm.
		9 - 24 months	
		22 - 33 lbs. weight 10 - 15 Kg.	29 - 37 in. length 74 - 94 cm.
		1 - 3 years	
		24 - 36 lbs. weight 11 - 16 Kg.	33 - 40 in. height 84 - 102 cm.
		2 - 5 years	
		26 - 42 lbs. weight 12 - 19 Kg.	35 - 45 in. height 89 - 114 cm.
		3 - 6 years	
		27 - 54 lbs. weight 12 - 25 Kg.	37 - 48 in. height 94 - 122 cm.
		5 years and up	
		weight 36 lbs. (16 Kg.) and up	height 42 in. (107 cm.) and up

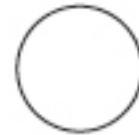

Please Refer to Pediatric Collar Instruction Sheet for more information.

© 2002 - 2004 Aspen Medical Products, Inc.
 (800) 295-2776 • (949) 681-0200
 www.aspenmp.com

983174 REV J 10/04

Aspen.

CERVICAL COLLAR SIZING GUIDE



Pick the Right Size

With the head in the neutral or desired position, place the guide on the highest point of the trapezius and flat against the side of the head.

Sight an imaginary perpendicular line from the bottom of the chin back to the guide and select the proper size collar.



Check for a Good Fit

- The chin must be flush with the front of the chin piece plastic.
- The FlexTabs™ on the front and back panels should be flexed.
- The collar should be "snug" but not touch the tracheal area.

Extra Tall

Tall

Regular

Short



Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.