

Mounting Instructions For Bed Use:

1. Choose location in center of headboard or on wall above center of headboard.
2. Drill holes for mounting screws (included), and attach bracket to headboard or wall.
3. Slide alarm into mounting bracket until it locks in place.
4. To remove alarm, push tab on bottom of bracket toward wall and slide alarm up and out of mounting bracket.

When using Sensor Pad as activation device:

1. Choose a location on footboard.
2. Drill holes for mounting screws (included), and attach bracket to footboard.
3. Slide alarm into mounting bracket until it locks in place.
4. To remove alarm, push tab on bottom of bracket toward wall and slide alarm up and out of mounting bracket.

For Chair Use:

Use metal mounting clip located on back of unit to attach alarm unit to wheelchair. Position to left or right of patient to avoid contact with patient's back. Optionally use short strap through clip and on to chair.

For seatbelt and Sensor only: Mesh Mounting Bag #75303 (sold separately) can also be used to securely mount alarm when used with a sensor pad or seatbelt.
Mesh Mounting Bag not recommended when using Magnetic Pull-Cord as activation device.

Reset Buttons

Dual reset buttons are located on each side of alarm. To reset when alarm sounds, press both reset buttons in unison.

Nurse-Call Jack

The VOICE Alarm is equipped with a nurse-call jack located on side of alarm unit. Plug one end of nurse-call cable #2035 (available separately) into alarm unit and other end into wall. When unit is activated, alarm will sound at alarm unit in addition to activating nurse-call system. Wall jack must also be a 1/4" stereo jack. Check with your facility's nurse-call system to ensure compatibility.

Warranty

The manufacturer guarantees the VOICE Alarm for a period of twelve months from date of purchase. This warranty is in force provided there is no evidence of misuse or improper installation.

To Reorder

#2032 VOICE Alarm w/Magnetic Pull-Cord

Accessories

#2035 Nurse-Call Cable w/Y-Adapter
#75042 Replacement Cords, 3/pk
#75303 Mesh Mounting Bag, 3/pk

Sensor Options:

Bed Pads
Belts
Floor Mats
Cushions
Chair Pads

Please visit AliMed.com/AliMed-sensors for a complete selection of sensor options.



Manufactured for:



AliMed® inc.

297 High St., Dedham, MA 02026 USA
PH 800-225-2610 • FAX 800-437-2966



MDSS GmbH
Schiffgraben 41
30175 Hannover,
Germany


AliMed.com

#5895REV0416 CCO824

©2016 AliMed® inc. All rights reserved.

Made in China

AliMed® IQ VOICE Alarm, #2032

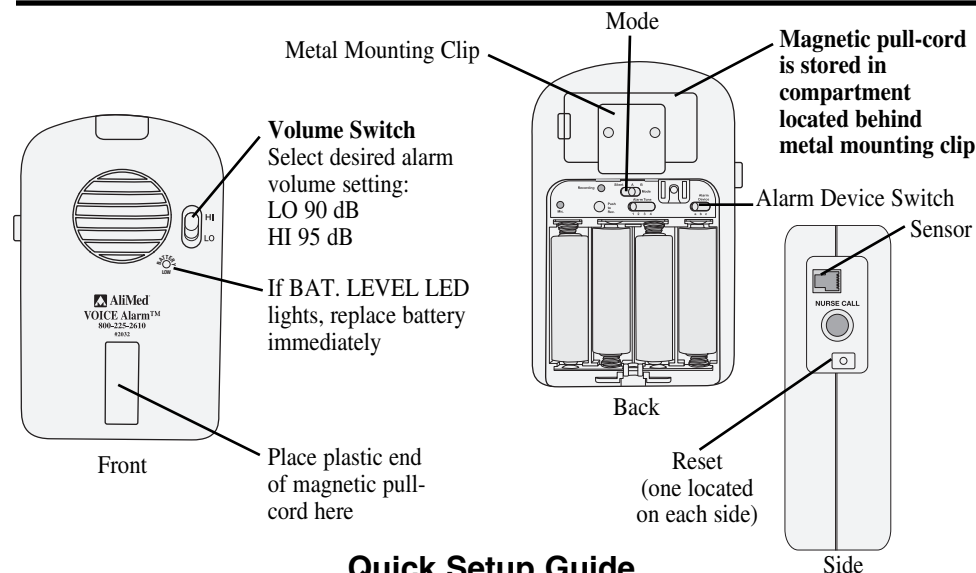

Quick Setup Guide

To use alarm feature only:

1. Loosen screw on back of battery cover and slide cover off.
2. Properly insert 4 AA batteries.
3. Mode select—Silent*, Voice, or Tone. See below for details.
4. Set switch to "Tone".
5. Set ALARM DEVICE switch to match activation device.
 - Setting "A"—Use Magnetic Pull-Cord clipped to patient's clothing (pull-cord located in storage compartment behind metal mounting clip).
 - Setting "B"—use Sensor Pad, Cushion Pad, Seatbelts, Universal Alarm Strap, or Lap Buddy.
 - Setting "C"—has no current application. Please DO NOT USE.
6. Select ALARM TONE (1-4) to sound.
7. Replace battery cover.
8. Select LO or HI volume setting.
9. This alarm is always on.

To Use Voice/Alarm Feature:

1. Loosen screw on back of battery cover and slide cover off.
2. Properly insert 4 AA batteries.
3. Mode select—Silent*, Voice, or Tone. See below for details.
4. To record personal message (maximum length 20 seconds):
 - a. Place magnetic pull-cord on front of alarm (pull-cord located in storage compartment behind metal mounting clip).
 - b. Set MODE switch to "Voice".
 - c. Set ALARM DEVICE switch to "A".
 - d. Remove magnetic pull-cord, and press and hold RECORD button. RECORD LED will light when unit is recording. Hold unit 8-10 inches from mouth to record.
 - e. Record message in a normal speaking voice, maximum length 20 seconds.
 - f. To stop recording, release RECORD button.
 - g. To replay message, return magnet to front of unit and remove to hear message.
 - h. To change message, repeat Steps a-g.
5. Follow Steps 5-8 in "Alarm Feature Only" instructions at left to complete setup.

*Silent Mode setting must be used with the nurse-call jack connection.

WARNING: Use of Silent Mode is for transmission of signal only. Will not audibly alarm locally.

Alarm Setup

1. Loosen screw on back of battery cover and slide cover off.
2. Insert 4 AA batteries, taking care to correctly orientate batteries.
3. Once setup is complete, slide cover back on and tighten screw.

Note: Locking mechanism for mounting bracket will not function properly without battery door cover screwed in place.

Select Alarm Mode

Select one of the following settings if the voice message feature of alarm will be active:

1. Voice/Alarm Mode—Set MODE switch to position “VOICE”

When unit is activated, voice recording will play for 20 seconds before changing over to alarm tone. If recorded voice message is less than 20 seconds, message will continue to replay until 20 seconds has elapsed, then alarm tone will begin to sound.

2. Alarm Tone Only—Set MODE switch to position “TONE”

When unit is activated, only selected alarm will sound.

3. Silent Mode—Can only be used with the nurse-call jack connection. This allows the alarm to only sound at the nurse-call station.

Record Voice Message (for Mode VOICE Only)

- The voice message you record may be up to 20 seconds in length and re-recorded as needed.
- Message is safely stored in memory and is not erased when batteries run low or are removed.
- To replace message, simply record new message in its place.

To record your personal voice message:

1. Remove magnetic pull-cord from storage compartment located behind metal mounting clip.
2. Place magnetic pull-cord on front of alarm unit.
 - Failure to use the magnetic pull-cord in the recording process can result in problems with recording.
 - If your message did not record properly or did not erase the previous message, record message again following these steps carefully.
3. Loosen screw on back of battery cover and slide cover off.
4. Remove magnetic pull-cord and press and hold RECORD button. RECORD LED will light when unit is recording. Hold unit 8-10 inches from mouth and in a normal speaking voice record message. Maximum message length is 20 seconds.
5. To stop recording, release RECORD button. Once you have exceeded 20 seconds, the unit will automatically stop recording and RECORD LED will turn off.
6. To replay message, return magnet to front of unit and remove to hear message.
7. To change your message, repeat steps 2-6.

Select Alarm Tone (1-4) to sound

Select Alarm Device

1. Set ALARM DEVICE switch to match activation device.
 - Setting “A”—Use magnetic Pull-Cord clipped to patient’s clothing (Pull-Cord is located in storage compartment behind metal mounting clip).
 - Setting “B”—Use Sensor Pad, Cushion Alarm, Seatbelts, Universal Alarm Strap, or Lap Buddy.
 - Setting “C”—Has no current application. Please DO NOT USE WITH ALIMED DEVICES.
2. When using Alarm Mode “B,” locate the cable socket on the side of the alarm unit and plug in selected device. Plug should “snap” into place. Gently tug the alarm wire to make sure the unit is connected properly and does not pull out from alarm unit. Do not continue use if phone jack in alarm unit fails to lock in place.

Note: Alarm will sound until device is attached.

3. When not in use, the magnetic pull-cord can be stored in the compartment located behind the metal mounting clip.

Use

Test alarm daily to ensure patient safety.

When using Magnetic Pull-Cord as activation device:

1. Mount alarm unit on chair or bed and place magnet on front of alarm unit.
2. Adjust cord length so that patient activates alarm once they begin to get out of bed or chair. Additional adjustments may be necessary based on individual patient.
3. Select appropriate volume level, LO or HI.
4. Ensure Alarm Device Switch under battery cover is switched to ‘A’.
5. Remove magnet from front of unit to test. Alarm will sound when magnet is removed. When magnet is returned to front of unit or when reset buttons are pressed in unison, alarm tone will stop sounding.
6. Low battery light will illuminate when it is time to change the batteries.
Note: If volume decreases or flashing Red LED light on top of unit becomes dim when alarm sounds, replace batteries immediately.

When using Bed/Chair Sensor Pad as activation device:

1. Locate sensor pad in bed or chair as recommended by manufacturer.
2. Plug sensor pad alarm cord into phone jack located on side of VOICE Alarm unit. Always check to ensure that plug is securely clipped into alarm unit.
3. Fasten any loose alarm wire to bed or chair to reduce risk of injury to staff or patient.
4. With weight applied to sensor pad, select the appropriate volume level, LO or HI.
5. Ensure Alarm Device Switch under battery cover is switched to ‘B’.
6. When weight is removed from sensor pad, alarm unit will sound. When weight is returned to sensor pad or when both reset buttons are pressed in unison, alarm tone will stop sounding.

When using seatbelt as activation device:

1. Fasten seatbelt to chair as recommended by manufacturer.
2. Plug seatbelt alarm cord into phone jack located on side of VOICE Alarm unit. Always check to ensure that plug is securely clipped into alarm unit.
3. Fasten any loose alarm wire to chair to reduce risk of injury to staff or patient.
4. With seatbelt buckled, select appropriate volume level, LO or HI.
5. Ensure Alarm Device Switch under battery cover is switched to ‘B’.
6. When seatbelt is unbuckled or hook and loop is separated, the alarm unit will sound. When belt is buckled or when reset buttons are pressed in unison, alarm tone will stop sounding.