





A wide selection for any indication

AliMed® Carbon Fiber AFOs

- **▶** Dynamic semi-flexible footplate
- ► Promotes natural gait pattern
- ► Enhances strength and stability to control torsion forces

Lightweight, modern, low-profile design provides added strength and stability while avoiding contact at critical pressure points and facilitating a normal walking pattern. Ideal for active clients.

The dynamic footplate and strut store and release energy to assist the patient through the gait cycle, adding strength with structurally reinforced high-stress areas. Footplates have 10°-15° of forefoot lift beginning around the metatarsal heads, yet still accommodate a foot orthotic. Open heel design helps eliminate pressure injuries. Flexible calf and tibial bands accommodate a wide range of calf circumferences. One-piece design with continuous strand does not excessively push out of the shoe. Ideal for patients with medial ankle instability, or that evert, pronate, or have valgus

Specify size (see chart on back cover) and Left or Right.

A. Posterior Lateral Strut

Spirals to the posterior with flexible calf bands. #66692

B. Anterior Lateral Strut

"Y" proximal anterior trimline relieves the tibial tuberosity and mid-tibia area.



AliMed® Swedish AFOs

- ► Lower profile than traditional plastic AFOs
- ► Enhanced airflow for cool comfort
- ► Heat-moldable and trimmable for customized fit

Delivers moderate lateral stability and static dorsiflexion assistance in a streamlined design for an easy, comfortable fit. Open calf and heel offers cool comfort and better shoe fit than a traditional plastic AFO. Trimmable footplate helps fit to any shoe. Ideal for a moderately active user.

Specify size (see chart on back cover).

C. FREEDOM® Swedish AFO

Injection-molded polypropylene • Padded hookand-loop calf strap • Trimmable footplate

wnite		ыаск	
#64328	Wmns Right	#64332	Wmns R
#64329	Wmns Left	#64333	Wmns Le
#64330	Mens Right	#64334	Mens Rig
#64331	Mens Left	#64335	Mens Le





Padded hook-and-

loop strap secures

around calf

Thicker

AliMed® Traditional Plastic AFOs

- ► Accommodates wide variety of needs
- ► Options for maximum-to-minimum medial/lateral stability
- ► Heat-moldable for customized fit

Reliable, economical options to meet a variety of needs. Can be applied in seconds, and easily modified by heat-molding or trimming for optimal customization. An ideal solution for short-term use.

See sizing chart on back cover.

D. Type C-90 This posterior leaf spring brace offers minimal medial/lateral stability and full plantarflexion with dorsiflexion assist. Ideal for users with less ankle instability.

#6603 Small, Left #6602 Small, Right #6605 Medium, Left #6604 Medium, Right #6607 Large, Left #6606 Large, Right

E. Type 670 This general purpose brace is trimmed high around the malleoli to provide significant medial/lateral stability and full plantarflexion control, and helps control positioning. Ideal for use as an acute post-CVA training brace. Typically requires heat-molding at malleoli.

#6673 Small, Left #6672 Small, Right

F. Type C-50 Provides moderate medial/lateral stability and plantarflexion control but is trimmed low in the malleolus area—less heat molding required. Ideal for general use or post-CVA use.

#6623 Small, Left #6622 Small, Right #6625 Medium, Left #6624 Medium, Right #6627 Large, Left #6626 Large, Right



Sliding cuff protects skin

G. Sliding AFO Sliding posterior cuff moves up and down. Protects against abrasion and maceration in calf. Padded cuff. Trim-to-fit footplate.

Specify size (see chart on back cover) and Left or Right.

#60748 Small #60749 Medium #60751 Large #60752 X-Large



For sizing information, see back cover

Specialty AFOs



Compact, attaches onto shoelaces and clips to an ankle cuff

H. FREEDOM® Adjustable Footdrop Brace

- Can be moved from shoe to shoe quickly and easily
- Ideal for individuals with mild to moderate footdrop

One size, see back cover for measuring instructions.

#66220 #66221

Adjustable Footdrop Brace Additional Shoe Straps, 3/pk



Encourages wearer to lift foot in preparation for swing phase

I. Intellinetix™ Step Sensor

- Uses vibration cells to provide reminder to lift foot while walking
- Lightweight, comfortable, discreet fits in any shoe
- Ideal for users with gait abnormalities associated with footdrop

One size.

#66665

Intellinetix Step Sensor



Ideal for the diabetic foot and other insensitive foot maladies

J. EZ Dorsi Assist AFO

- Dynamic carbon fiber construction is lightweight but strong
- Adjustable height and flexible calf band eliminate all fitting issues
- Installs in less than 30 minutes, all hardware included

Specify size (see chart on back cover).

#66698

EZ Dorsi Assist AFO



Can be worn barefoot, with sandals, or slippers

K. FREEDOM® Soft Footdrop Brace

- Flexible solution for when regular footdrop braces cannot be worn
- Semirigid plantar footplate maintains arch
- Ideal for hypermobile ankle or post-polio afflictions

Specify size (see chart on back cover).

#64105 Soft Footdrop Brace, Small #64106 Soft Footdrop Brace, Medium #64107 Soft Footdrop Brace, Large



Dorsiflexion-assist Jacob Joint™

L. Step-Smart[™] Brace

- Pushes the foot into dorsiflexion during swing phase
- Absorbs shock and decelerated plantarflexion during heel strike
- Clinician can adjust shock absorption level

Specify size (see chart on back cover).

'0583 Step-Smart Brace



For younger patients

M. Pediatric AFO

- Allows immediate fitting for early ambulation
- Stabilizes ankle with 1/8" polypropylene shell for medial/lateral support
- Helps resist spastic tone

Specify size (see chart on back cover).

#66131 Pediatric AFO, Right #66132 Pediatric AFO, Left



NEW! eCatalogs

Now online with live links!

AliMed.com/ecatalog

AFO Sizing Information



A-B. Carbon Fiber AFO

AFO Size		Height Anterior	Malleolus Relief Range*	Footplate Length
X-Small	13"	14"	21/2"-31/2"	9"
Small	14"	15"	3"-4"	10"
Medium	15"	16"	3"-4"	11"
Large	16"	17"	31/2"-41/2"	12"
*Measure	floor to ank	le bone.		



c. FREEDOM® Swedish AFO

Wmns fits up to size 91/2; Mens fits up to size 12.



D-F. Traditional Plastic AFO— Type C-90, Type 670, Type C-50

11¾"
12"
123/4"



G. Sliding AFO

AFO Size	Shoe Size	AFO Height
Small	Wmns 3-6/Mens 5-7	13¾"
Medium	Wmns 6-9/Mens 7-9	13¾"
Large	Wmns 9-11/Mens 9-11	14"
X-Large	Wmns 11+/Mens 11-13	15¾"



H. FREEDOM® Adjustable Footdrop

Brace One Size—Measure circumference of leg just above ankle at narrowest point. Fits up to 10½".

AliMed.com

Prices, product availability, and specs are subject to change.



I. Intellinetix[™] Step Sensor



J. EZ Dorsi Assist AFO

AFO Size	Calf Band Circumference 8"-10"		
X-Small			
Small	11"-13"		
Medium	14"-16"		
Large	17"-19"		



K. FREEDOM® Soft Footdrop Brace

AFO Size	Shoe Size	
Small	Wmns 6-7½	
Medium	Wmns 8-10½/Mens 7-9½	
Large	Wmns 11/Mens 10-12	



L. Step-Smart[™] Brace

AFO Size	Shoe Size	Length
Small/Medium	Wmns 7-12	
	Mens 5½-10½	61/4"
Large/X-Large	Wmns 12½-14	
	Mens 101/2-121/2	6¾"



M. Pediatric AFO

AFO Size	Calf Circumference	Brace Height	Foot Length
X-Small	6"-71/2"	61/2"	4"
Small	71/2"-9"	7"	41/2"
Medium	81/2"-91/2"	9"	43/4"
Large	9"-101/2"	10"	51/4"
X-Large	10"-11½"	11"	6"

Pub. #7225REV0921 Product #11207 ©2021 AliMed® Inc. All rights reserved.

