

Proper care extends the life of your aprons

To ensure the longevity of your new garment, please follow these simple guidelines:

- Radiograph your garment upon receipt and regularly after that date to ensure no damage has occurred to the protective core.
- Hang your apron properly either by the shoulders or straps.
- Do not fold or crease the garment.
- Hook-and-loop fasteners should be matched together so they do not snag or tear the fabric.
- Do not expose the garment to extreme temperatures; submerge in liquids or place in dryers.
- Do not lean or rub against sharp corners/edges or store sharp objects in the pockets.

Cleaning

Clean your protective wearables in accordance with your facility's infection control protocols.

Remove stains immediately when they occur. If this is not possible, clean with warm water and a non-bleach dishwashing soap. Do not use harsh abrasive cleaners. Some upholstery cleaners that remove protein stains may be used to clean blood.

Warranty

AliMed guarantees its aprons against defects in materials and workmanship for two years from date of factory shipment.

SMARTID[®]

SmartID is a web-based inspection and inventory asset management system. Visit AliMed.com/smartid for more information.



Read more on the blog!

*Extending the Life of Your
Radiation Protection Aprons*

[Learn more](#)

Apron Care Products

■ Brush away tough stains

Scrubbles Apron Cleaner

Enzyme-active formula cleans barium, blood, and other common radiology-related stains from protective aprons. Simply spray it on, let it soak, and brush.

32-oz. spray bottle with brush

#937910



■ Fast-acting surface wipes

CaviWipes™ These highly effective, fast-acting disinfectant surface wipes with CaviCide[®] fight viral, bacterial, and fungal contamination. Wipes kill TB in three minutes and MRSA, HIV1, HBV, and HCV in just two minutes. 160 wipes per canister, 12 canisters/cs.

#935997

 **Hazardous materials must ship ground.**



■ No-slip hangers

Heavy-Duty Apron Hangers

are ideal for use with valet racks or in closets to keep aprons off floor. Lifted sides prevent garment slipping. 3/pk.

³/₁₆"-diameter welded steel rod
• Chrome-plated

#937056

