

ROUTE TO:

- Director of Nursing
- Director of Rehab
- Purchasing Director
-

CUSHIONS

for **all patients**

Standard-use cushions to stock
Specialty cushions for unique needs



*How to
choose the
right cushion...*

Patient comfort and protection is essential in any facility...

(cont. on p2)

Buying cushions for a facility?
Not sure what to stock?

Wheelchair cushion

Foam Comfort Cushions

These rectangular blocks of foam are covered with either a fabric (preferably knit) or wipe-clean, waterproof covers. A four-way stretch-knit fabric cover offers a cooler sitting surface with less interference with the basic properties of the foam.

However, incontinence and spillage often make a waterproof cover more useful in a facility.



AliMed offers three grades of quality foam cushions: **Value Line**, **HR (High Resilience)**, and **Latex**. The Value Line offers good overall performance. The HR polyurethane foam cushion conforms well for enhanced comfort and is highly durable, yet latex-free. Latex foam, in the opinion of many, has never been equaled in terms of long-term sitting comfort.

Comfort and Skin

Protection Cushions When managing large populations of fragile-skinned, sedentary individuals, consider the option of superior pressure distribution. The better a cushion conforms to the sitter's anatomy, the lower the pressure experienced. T-Foam or temper-foam, was the original "memory foam" offered, and still ranks among the best. T-Foam is able to mold and contour to subtle



anatomic features, resulting in lower, more uniform pressure distribution. Memory foam, when used on its own, must be matched to the user's weight to avoid bottoming out. For general facility use, our T-Foam Combo is preferred because it is not weight-dependent. It consists of three layers to offer instant comfort, long-term support, and a firm base to prevent bottoming out.

Another method to achieve pressure-lowering uniformity is by dividing foam into individual blocks that act independently, like a series of many independent small cushions. The surface of our **Independent Cell Cushion** has individual 2" squares that act as springs, adjusting to anatomical contours, minimizing pressure over bony prominences and shifting independently to reduce friction and shear.

Foam Comfort Cushions
pp4,5

Comfort and Skin Protection
Cushions pp6,7



Let us help you!

Patient comfort and protection is essential in any facility. That is why careful attention must be paid to wheelchair cushion selection. This Cushion Selection Guide will help you choose the right cushion for your patient's needs.

You must first categorize your cushion-buying needs. There are **Facility Cushions**—these are your standard-use cushions that offer comfort and basic cushioning for the general patient population. These can be divided into several groups: Standard Foam Comfort Cushions, Comfort and Skin Protection Cushions, Comfort and Positioning Cushions, and Gel Cushions. Conversely, there are **Specialty Cushions** that meet the specific needs of an individual, or a smaller group of users, whether for fragile skin protection, postural support, security, or any combination of these

Also, see our section on Fall Prevention Cushions, p. 42.

selection made easy

Comfort and Positioning

Cushions A common problem for wheelchair users is sacral sitting—the act of sliding forward on the seat and going into a posterior pelvic tilt. After several minutes this becomes extremely uncomfortable, primarily because of the high degree of pressure placed on the sacrum. In addition, sacral sitting compromises the respiratory function as well as the ability to perform normal tasks like eating.

A wedge cushion, with the high side at the chair front, can help. It keeps the hips thrust back in the chair, promoting an anterior pelvic tilt, and also keeps the back in contact with the backrest. The result is upright posture and improved comfort, sitting tolerance, and function.




In order to be effective, the wedge must be of high-quality foam construction that will not collapse under the sitter's weight. AliMed's simple wedge is firm enough to maintain its shape, yet comfortable for all-day sitting.

Alternatively, for general use, consider the **Sit Straight GP**. It incorporates a gentle wedge, an ischial dish to reduce sacral pressure, and a convex base that fills the wheelchair sling-seat sag. Filling the sling-seat sag maintains a flat sitting surface, much the same as with a solid-seat insert. This is a preferred sitting surface—that stabilizes the sitter and avoids unwanted pressure points.

Comfort and Positioning
Cushions, pp8,9

Gel Cushions The outstanding characteristic of gel cushions is their ability to absorb shear (friction) forces in addition to giving moderate pressure distribution. Shear is the result of sliding and slipping on a sitting surface.

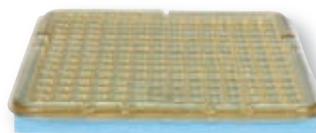

Gel, however, acts like an additional layer of fatty tissue, absorbing destructive shear forces.

Two kinds of gels are commonly used in wheelchair cushions—elastomeric and water-based. An elastomer is a soft, viscoelastic solid rubber material that redistributes heat and weight across the cushion's surface. Water-based gel, however, is a mixture of water and a substance to give it a syrupy consistency. Water-based gels must be contained in a heavy leak-proof bladder.

Clearly the elastomer gel cushions are more effective for skin protection, more reliable, and maintenance-free.


Gel Cushions
pp10,11

Specialty Cushions for

specific needs There are a host of cushions that require an analysis of the unique needs of the patient to determine the appropriateness of a particular cushion.



AliMed offers an extremely wide selection of specialty cushions, including specialty foam, gel, and air-cell options, wedge, pommel, anti-thrust and multipurpose positioning cushions. We have cushions for specific diagnostic groups including stroke, hip replacement, and amputee. We offer many cushions in bariatric sizes

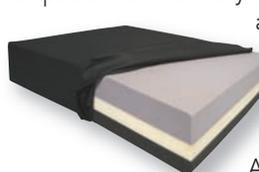

as well. In addition to the many AliMed cushions, we also offer cushions from Jay, ROHO, Relax, Action, Keen, and other

leading manufacturers. And if you just can't seem to find the right cushion for your specific needs, call us! We can make a custom cushion to your specific requirements.



Specialty Cushions
start on p12

Foam Comfort Cushions. Affordable Prices. Equip Your Facility.



Foam Comfort

Cool, comfortable cloth cover with heat-sealed seams



Incontinence-proof vinyl cover



Priced right



General-purpose Economy Cushions

are basic foam seat cushions that can be used on a wheelchair or bedside chair to improve comfort and sitting tolerance. They are typically made of soft, spongy foam—similar to upholstered furniture—and are recommended for any user without skin integrity issues. These cushions are 17"W x 16"D, so they will fit on a 16"W or 18"W wheelchair—no need to stock multiple widths!

Choose a cover: Cloth covers are non-removable, incontinence resistant and cool and comfortable against skin. Wipe-clean vinyl covers are incontinence-proof and very durable.

Cloth Cover, 12/cs

Item #	Height	Case Price	Buy 3 or more cases
#EPP1934	2"	\$217.00 case Only \$18.08 each	\$196.00 case Only \$16.33 each
#EPP1935	3"	290.00 case Only \$24.17 each	263.00 case Only \$21.92 each

Vinyl Cover, 12/cs

Item #	Height	Case Price	Buy 3 or more cases
#EPP1936	2"	\$310.00 case Only \$25.83 each	\$290.00 case Only \$24.17 each
#EPP1937	3"	333.00 case Only \$27.75 each	294.00 case Only \$24.50 each

Sit longer, comfortably!

High-resilience Foam Cushions These latex-free, high-resilience foam cushions offer all-day comfort and resist bottoming out. Springy foam conforms well to the user, cradling anatomical contours. 3"H, covered in incontinence-proof, rip-stop nylon cover. 12/cs.



Latex-free foam



Incontinence-proof nylon cover

Superior comfort

Latex Comfort Cushions Latex foam offers the utmost in sitting comfort. The high resiliency of this soft and springy foam provides all-day comfort for even the most particular of users. Because of the inherent elastic, yet firm qualities of rubber latex, cushions made of latex foam conform to the body and maintain their shape over time. In addition, latex won't absorb moisture and naturally resists the growth of bacteria, mold, and mildew, making it a great choice for the user with occasional incontinence.

Available in 2"H, 3"H, or 4" thicknesses. Removable black knit, open-weave cover enhances comfort and breathability. 18"W x 16"D. 12/cs.



Latex offers unbeatable comfort

High-resilience Foam Cushions, 12/cs

Item #	W x D	Case Price	Buy 3 or more cases
#EPP1107012	18" x 16"	\$390.00 case Only \$32.50 each	\$360.00/case Only \$30.00 each
#EPP1106912	16" x 16"	390.00 case Only \$32.50 each	360.00/case Only \$30.00 each

Latex Comfort Cushions, 12/cs

Item #	Height	Case Price	Buy 3 or more cases
#EPP14012	2"	\$387.00 case Only \$32.25 each	\$327.00 case Only \$27.25 each
#EPP14112	3"	513.00 case Only \$42.75 each	453.00 case Only \$37.75 each
#EPP14212	4"	594.00 case Only \$49.50 each	534.00 case Only \$44.50 each

Cushions for Comfort and Skin Protection



Uniform pressure, shear reduction—easy setup!

Independent Cell Cushion Air-cell cushions have long been touted as the leader in shear and pressure relief, but the high-cost, complex setup, and ongoing maintenance needs, makes them a poor choice for a general-purpose facility cushion. By building an independent-cell foam cushion, we have created a low-cost, easy setup product with many of the pressure- and shear-reducing attributes of an air cell cushion. The 70+ foam cells offer uniform pressure distribution and are able to shift independently as the user moves reducing friction against fragile skin. The cells act as springs, adjusting to anatomical contours and minimizing pressure over bony prominences. Spaces between cells also allow air circulation and heat reduction. Cushions are 4" H with incontinence-proof, rip-stop nylon cover. 6/cs.

Lower profile, plus added pressure relief

Checkerboard T-Foam™ Combo Cushion Independent-cell top layer with a self-molding, shock-absorbing viscoelastic T-Foam base (medium-resistance) for greater pressure relief at ischial tuberosities and coccyx—all in a lower profile cushion. 3" H. Black polyester knit cover. 6/cs.

Independent Cell Cushion, 6/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP11 7006	18"W x 16"D	\$265.50 case Only \$44.25 each	\$235.50 case Only \$39.25 each
#EPP11 7106	16"W x 16"D	297.00 case Only \$49.50 each	267.00 case Only \$44.50 each

Checkerboard T-Foam Combo Cushion, 6/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP12 1706	18"W x 16"D	\$398.00 case Only \$66.33 each	\$358.20 case Only \$59.70 each
#EPP12 1806	16"W x 16"D	398.00 case Only \$66.33 each	358.20 case Only \$59.70 each

Comfort and pressure relief—without weight restrictions!

AliMed T-Foam™ Combo Cushion This triple-layer foam cushion offers the best of all worlds: a top layer of conforming soft pink T-Foam, a middle layer of pressure-relieving blue medium T-Foam, and a base layer of high-density foam that helps prevent bottoming out, even at higher weights. A true “universal” T-Foam cushion, perfect for outfitting a facility when skin integrity issues are a concern. Requires a flat sitting surface. 3”H. Black AliTex™ top cover is a water-resistant stretch fabric that “breathes” for added comfort. Non-slip vinyl bottom helps keep cushion in place on wheelchair. 6/cs.



Separate layers allow for custom modification


AliTex cover “breathes” for comfort

T-Foam Combo Cushion, 6/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP11 7606	18"W x 16"D	\$561.60 case Only \$93.60 each	\$505.44 case Only \$84.24 each
#EPP11 7506	16"W x 16"D	561.60 case Only \$93.60 each	505.44 case Only \$84.24 each

AliMed® T-Foam™ Cushions for Optimal Pressure Relief


T-Foam helps reduce painful pressure points and molds to body's contours to ensure optimal pressure distribution.

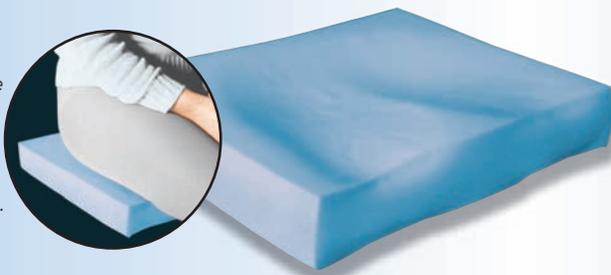

What sets a T-Foam Cushion apart from other foam wheelchair cushions is the phenomenon called “slow recovery.” When a person sitting on a T-Foam Cushion shifts weight or lifts their body even slightly, T-Foam retains its compressed shape long enough to allow the person’s capillary vessels to nourish tissues on the weight-bearing surface. The T-Foam then “recovers” its original shape, reducing pressure points and helping prevent pressure sores.

Available in:

Standard Foam Cushions, p18

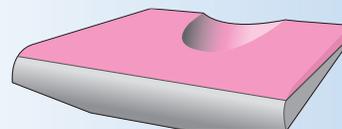

Positioning Cushions, p16

Gel-Foam Versions, p20



Cushions for Comfort and Positioning



How do wedge cushions help improve posture?

A wedge cushion keeps the user from sliding forward in a chair by positioning the pelvis in an anterior pelvic tilt with the knees slightly higher than the hips. This position naturally encourages the spine to maintain a more upright posture, and the slope of the cushion encourages the buttocks to remain back on the seat cushion. This decreases pressure on the sacrum and coccyx, and encourages improved breathing, better head control, and better positioning for self feeding.



Wipe-clean cover extends durability

High-density foam core cushions and supports without bottoming out.



Basic Wedge Cushion This simple wedge design creates a comfortable sitting surface that discourages slouching and maximizes sitting tolerance. High-density foam contours to body for added support, yet holds its shape even with long-term use. Cushion tapers from 3" H in front to 1" H in rear.

Cover options: Choose between two removable covers. **Nonslip Vinyl Cover** is a wipe-clean fabric that further limits forward sliding and is great for long-term durability. **Knit Cover** is a washable polyester fabric that enhances comfort and reduces heat buildup.

Vinyl Cover, 12/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP26012	18"W x 16"D	\$402.00 case Only \$33.50 each	\$342.00 case (3+) Only \$28.50 each
#EPP26112	16"W x 16"D	402.00 case Only \$33.50 each	342.00 case (3+) Only \$28.50 each

Knit Cover, 12/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP10 8812	18"W x 16"D	\$378.00 case Only \$31.50 each	\$318.00 case (3+) Only \$26.50 each
#EPP10 8912	16"W x 16"D	378.00 case Only \$31.50 each	318.00 case (3+) Only \$26.50 each



All the benefits of a wedge, plus...

- Limits slouching and forward sliding
- Creates a steady, flat seating surface
- Improves posture comfort
- Softer foam offers greater all-day comfort
- Maximizes sitting tolerance
- Adds extra skin protection



More than just a wedge cushion...Only \$39.99 each!

 **Sit-Straight™ GP Wedge** delivers all the benefits of a basic wedge cushion, but it doesn't stop there! The convex base fills the sag of a traditional sling seat, creating a flat sitting surface that won't tip, fold, or buckle. Constructed of a softer, suppler foam that brings unsurpassed comfort to an everyday, general-purpose cushion. The sculpted base, gentle slope, and scooped coccyx cutout offer added skin protection. For an all-day comfort cushion that won't bottom out, choose the new Sit-Straight GP Wedge!

Dimensions: 18"W x 16"D or 16"W x 16"D • 3"H front slopes to 2"H rear
 • Polyfoam core • Nonslip Vinyl or Knit Cover options, both with rear ties



Cover options: Two cover options meet the needs of any facility. Durable, wipe-clean **Nonslip Vinyl Cover** with waterfall zipper is ideal for populations with heavy incontinence issues. Its nonslip surface helps reduce forward sliding, and is removable for simple replacement if needed. **Stretch Knit Cover** option is removable for laundering, offers cooling comfort, and is gentle on skin.

Nonslip Vinyl Cover, 12/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP11 04712	18"W x 16"D	\$479.88 case Only \$39.99 each	\$419.88 case (3+) Only \$34.99 each
#EPP11 04612	16"W x 16"D	479.88 case Only \$39.99 each	419.88 case (3+) Only \$34.99 each

Knit Cover, 12/cs

Item #	Size	Case Price	Buy 3 or more cases
#EPP11 04512	18"W x 16"D	\$479.88 case Only \$39.99 each	\$419.88 case (3+) Only \$34.99 each
#EPP11 04412	16"W x 16"D	479.88 case Only \$39.99 each	419.88 case (3+) Only \$34.99 each

Gel Cushions for Skin



What's in a Gel?

Gel cushions have long been touted for skin protection and comfort. They minimize heat, improve sitting tolerance, and are easily wiped clean. But not all gel cushions are created equal. There are two basic types of gels used in wheelchair cushions: aqueous (water-based) and elastomer (rubber-like) gels.

Aqueous Gels are viscous, must be encased in a plastic bladder, and typically cost less than elastomer gels. Aqueous gel offers cooling comfort and, when coupled with foam, helps to redistribute pressure away from bony prominences. Aqueous gels do not offer the same shear relief as elastomer gels and can bottom out. Water-based gels can also leak and dry out if punctured.

Elastomer Gels are solid sheets of soft, rubber-like polymer that mimic fatty tissue. Elastomer gel moves with the skin to minimize the effects of skin-shearing and the risk of skin breakdown. Used alone, it offers shear relief and impact absorption. Checkerboard versions offer even greater air circulation to fragile skin. Our T-Gel elastomer gel is unconditionally guaranteed not to leak or dry out if accidentally punctured or cut. An elastomer gel pad can be layered onto a viscoelastic memory foam pad for even greater pressure relief and positioning.



Super-Econo-Gel Wheelchair Cushion is a great general-purpose gel cushion for your facility needs. It combines an aqueous gel bladder core and cushioning foam outer layer to provide comfort and skin protection for the wheelchair user. Incontinence-resistant, washable nylon cover with nonslip bottom. 3" (½" gel bladder between two 1¼"-thick foam sections), 250-lb. capacity.



Econo-Gel, 12/cs

Item #	Size	Case Price	Buy 2 or more cases
#EPP1272	18"W x 16"D	\$478.00 cs Only \$39.83 ea	\$466.00 cs Only \$38.83 ea
#EPP1273	16"W x 16"D	478.00 cs Only \$39.83 ea	466.00 cs Only \$38.83 ea

Suggested code: E0192

Protection and Comfort

T-Gel™—optimal shear reduction.

T-Gel is an **elastomer gel**—a solid sheet of soft, rubber-like polymer that mimics fatty tissue. As with typical cushions, excessive shear force leads to occlusion of blood flow, which is seen as one of the biggest contributing factors behind pressure sores and discomfort.


But T-Gel moves with the skin to minimize the effects of skin-shearing and the risk of skin breakdown. It is guaranteed not to leak or dry out if accidentally punctured or cut.

Choose between two gel options:

Smooth—
Basic shear relief.
Minimizes the effects of skin shearing with smooth surface.



Checkerboard—
Maximum shear relief. Each individual gel cell can move independently to react to the user's movements and allow greater air circulation to fragile skin.

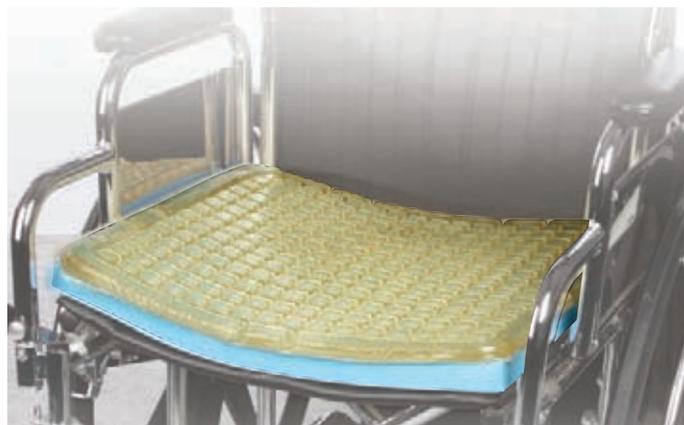


T-Gel™ and T-Foam™ combined to reduce shear and redistribute pressure.



Checkerboard

Smooth

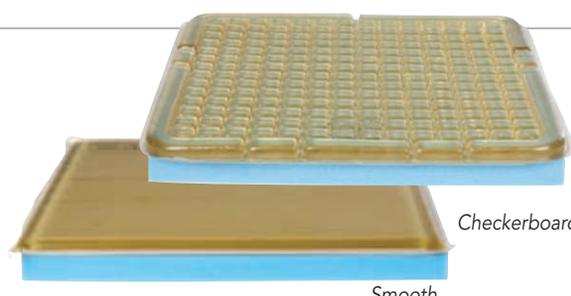

Checkerboard

Smooth

T-Gel™ Cushion

- Reduces shear

Excellent for applications that require minimal thickness. T-Gel polymer gel “slides” laterally, effectively reducing harmful shear forces and skin-damaging friction. Low-profile design allows use on any sitting surface. Black stretch knit cover. Smooth: 5/8" H. Checkerboard: 3/4" H. 5 1/2 lbs. each.

T-Gel Cushion: Smooth

#EPP1321	16"W x 16"D	\$129.00 ea	Only \$105.00 ea (6+)
#EPP1319	18"W x 16"D	129.00 ea	Only \$105.00 ea (6+)
#EPP10874	20"W x 18"D	160.00 ea	Only \$124.00 ea (6+)

T-Gel Cushion: Checkerboard

#EPP1530	16"W x 16"D	\$161.00 ea	Only \$130.00 ea (6+)
#EPP1529	18"W x 16"D	161.00 ea	Only \$130.00 ea (6+)
#EPP10882	20"W x 18"D	176.00 ea	Only \$140.00 ea (6+)

Buy more and save!

T-Gel™ Plus Cushion

- Reduces shear
- Redistributes pressure

Combines the benefits of T-Gel with those of slow-recovery T-Foam for even better pressure distribution and shear reduction. Medium-resistance T-Foam layer magnifies the positive effects of the gel by offering extra cushion to evenly allocate coccyx pressure. Alitane® envelope protects the T-Foam from incontinence. Black stretch knit cover. Smooth: 1 5/8" H. Checkerboard: 1 3/4" H. 8 lbs. each.

T-Gel Plus Cushion: Smooth

#EPP1293	16"W x 16"D	\$172.00 ea	Only \$140.00 ea (6+)
#EPP1292	18"W x 16"D	172.00 ea	Only \$140.00 ea (6+)
#EPP10877	20"W x 18"D	201.00 ea	Only \$165.00 ea (6+)

T-Gel Plus Cushion: Checkerboard

#EPP1530	16"W x 16"D	\$161.00 ea	Only \$130.00 ea (6+)
#EPP1529	18"W x 16"D	161.00 ea	Only \$130.00 ea (6+)
#EPP10882	20"W x 18"D	176.00 ea	Only \$140.00 ea (6+)

Buy more and save!

Cushions for a specific need



Specialty Cushions


Individual cushions for your specific needs

An analysis of the unique needs of a patient will determine the appropriateness of a particular cushion. AliMed offers an extremely wide selection of specialty cushions, including oversized (bariatric) cushions, and we have the ability to make custom cushions to your specifications if necessary.

Throughout this section look for our most popular brands of cushions:


From AliMed...

- T-Foam (Temper Foam)
- T-Gel
- Sit-Straight
- New! Stay-Put™
- AeroCell II

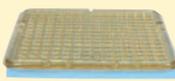

And...

- ROHO
- Jay
- Relax
- Skil-Care
- Posey
- Keen
- Action
- Star

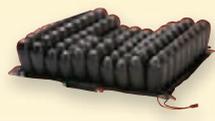

...and many more

Custom Cushions... to *your* specifications

Despite the wide array of cushions we offer, sometimes you just can't find exactly what you are looking for. Whether it is an odd size you need, or a completely unique cushion configuration, AliMed can make it for you. We offer custom sizes and shapes, specialty cushion toppers, and cutouts to accommodate anatomical deformities, as well as custom or replacement covers.



Wheelchair Cushions



Custom shape or size, even oversized!

Replacement Covers



Add a T-Foam or T-Gel topper to any cushion



Replace a cover, not a cushion

Back Supports



Custom cutouts

Any cushion or wedge—not sure what you need?

Our helpful and knowledgeable Product Support Team is just a phone call away.

Call 800-225-2610 x 103

Replace those old cushion covers!

Replacement Cushion Covers are handy when washing your cushions' original covers and when it's time to replace your worn-out covers. Select a fabric from the list below and indicate width/depth and thickness.

Select fabric:

- #EPP165Z Stretch Knit, Black.....\$44.00 ea
- #EPP165ANS AliTex/No-Slip Bottom, Black top, Grey bottom.....44.00 ea
- #EPP165W Naugahyde Cover, Black44.00 ea

Choose width/depth:

16"W x 16"D, 18"W x 16"D, 18"W x 18"D, 20"W x 18"D, 22"W x 18"D.

Choose thickness*: 1", 2", 3", 4" or custom-order thickness at no extra charge.

*Custom sizes available at no additional charge. Call 800-225-2610 x 103.

Customized cushions cannot be returned.



Wedge Cushions



Wedge cushions promote upright sitting

Wedge cushions are used to help prevent a wheelchair user from sliding forward in the chair (sacral sitting). The risks associated with sacral sitting include skin breakdown at the coccyx and thoracic regions, falls from the wheelchair, poor sitting tolerance, and loss of functional abilities. A wedge cushion helps by positioning the pelvis in an anterior tilt with the knees slightly higher than the hips. Wedge cushions can also be used to keep users' hips properly positioned at 90° when seated in a reclined position.

Basic Wedge

as low as
\$38.75
ea



Basic Wedge High-density polyurethane foam maintains shape. 3"-high front tapers to 1" rear. Choose incontinence-proof, upholstered vinyl nonslip cover or polyester stretch knit cover for greater shear control.

Vinyl Upholstered Cover


- #EPP260 18"W x 16"D\$46.00 ea...\$38.75 ea (10+)
- #EPP261 16"W x 16"D46.75 ea.....38.75 ea (10+)
- #EPP1287 14"W x 14"D67.00 ea.....58.97 ea (10+)

Polyester Knit Cover

- #EPP1088 18"W x 16"D\$46.00 ea...\$39.97 ea (10+)
- #EPP1089 16"W x 16"D64.75 ea.....57.75 ea (10+)

Sit-Straight™ with T-Gel™ Cushion

With an added top layer of checkerboard T-Gel that offers shear reduction and pressure redistribution. T-Gel's elastomer gel mimics fatty tissue and moves with the skin to minimize shear and skin breakdown. Tapered front edge relieves pressure on thighs. Comes with wipe-clean, water-resistant vinyl cover with nonskid bottom.

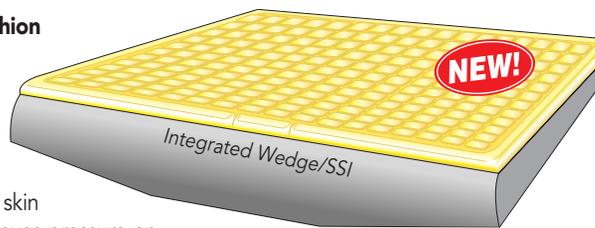

- #EPP11009 18"x16"\$167.75 ea\$161.75 ea (6+)
- #EPP11010 16"x16"170.75 ea164.75 ea (6+)

➔ **More Sit-Straight™ Wedge Cushions**, pp16, 17



Super Wedge Cushion Individual cells distribute pressure better than a flat surface. Ischial cutout on the underside reduces sacral pressure. Wedge shape promotes upright posture.

- 18" x 16" • 2½" back; 4" front • Ripstop nylon cover is liquid-resistant
- #EPP1172\$72.75 ea\$61.75 ea (10+)



Foam Wheelchair Wedge Incontinence-proof vinyl wipes clean. 18"W x 16"D.

- #EPP8493 4" Front, 2" Back\$39.75 ea
- #EPP8494 6" Front, 3" Back42.75 ea



Convex Gel Wedge Foam encases a pressure-reducing gel bladder. Convex bottom for stable, level seat.

- Nonslip, water-repellent vinyl cover zips off • 2" to 4" thick • 18"W x 16"D
- #EPP1350\$95.75 ea



Ultra-Wedge fills sling seat to create flat, stable sitting surface. High-density foam or gel bladder top layer.

- Incontinence-proof vinyl cover • Rear safety ties • Antislip bottom • 18"W x 16"D • Foam only: 3½" front, 1½" rear • Gel/Foam: 4" front, 1½" rear

- #EPP8495 Foam only\$ 69.75 ea
- #EPP8782 Gel/Foam109.00 ea

Wedge

Foam Utility Cushions

For sitting comfort and tolerance



Latex Wheelchair Cushion High-quality foam provides natural, supportive comfort with its firm material. Choose 2", 3", or 4"H. Choose between Black Knit, open-weave cover for comfort and aeration, or Blue Sure-Check® incontinence-resistant cover.

Black Knit

#EPP140	18"W x 16"D x 2"\$45.00 ea.....	\$39.00 ea (6+)
#EPP141	18"W x 16"D x 3"62.00 ea.....	55.00 ea (6+)
#EPP142	18"W x 16"D x 4"70.00 ea.....	63.00 ea (6+)
#EPP140-JR	16"W x 16"D x 2"44.00 ea.....	38.00 ea (6+)
#EPP141-JR	16"W x 16"D x 3"62.00 ea.....	55.00 ea (6+)
#EPP142-JR	16"W x 16"D x 4"70.00 ea.....	63.00 ea (6+)

Blue Sure-Check

#EPP1095	18"W x 16"D x 2"\$55.00 ea
#EPP1097	18"W x 16"D x 3"62.00 ea
#EPP1096	16"W x 16"D x 2"55.00 ea



Iris® Cushion Dimpled surface and contoured base enhance comfort. Wedge design limits forward sliding. Multiple foam layers for weight distribution and support. 4"H front, 3"H rear. Vinyl cover. 18-month guarantee. 18"W x 16"D. #EPP62504\$64.75 ea.....\$60.75 ea (3+)



T-Foam helps reduce painful pressure points; molds to body's contours to ensure optimal pressure distribution.



T-Foam™ Utility Cushion Conforms and molds with body heat and pressure. Redistributes weight evenly for the patient. Black polyester knit cover. 3"H.

#EPP164M	18"W x 16"D.....	\$99.00 ea
	86.75 ea (6+)
#EPP164M-JR	16"W x 16"D.....	99.00 ea
	86.75 ea (6+)



Polyfoam Utility Cushion This cushion offers sitting comfort with its lightweight, soft, spongy foam. Choose 2" or 3" thickness. Black, waterproof, bacteriostatic, flame-retardant, reinforced vinyl cover with rear ties.

#EPP1091	18"W x 16"D x 2"	\$45.75 ea
	41.75 ea (3+)
#EPP1093	18"W x 16"D x 3"	53.00 ea
	47.22 ea (6+)
#EPP1092	16"W x 16"D x 2"	42.00 ea
	40.75 ea (6+)
#EPP1094	16"W x 16"D x 3"	60.00 ea
	46.00 ea (6+)



Urethane Foam Utility Cushion Higher density than polyfoam to reduce risk of bottoming out. Choose 2" or 3" thickness. Black or Navy Naugahyde® wipe-clean vinyl cover.

#EPP1253	18"W x 16"D x 2"	\$52.00 ea
	48.00 ea (6+)
#EPP1255	18"W x 16"D x 3"	57.00 ea
	52.00 ea (6+)
#EPP1252	16"W x 16"D x 2"	52.00 ea
	48.00 ea (6+)
#EPP1254	16"W x 16"D x 3"	57.00 ea
	52.00 ea (6+)

Specify Black or Navy.



Checkerboard T-Foam™ Cushion Combo
 Independent-cell top layer with a self-molding, shock-absorbing viscoelastic T-Foam™ base (medium-resistance) for greater pressure relief at ischial tuberosities and coccyx in a lower profile cushion. 3"H.

- #EPP1217 18"W x 16"D\$90.25 ea\$86.75 ea (3+)
- #EPP1218 16"W x 16"D87.75 ea83.75 ea (3+)

Pressure and shear reduction

Cells adjust for uniform pressure distribution. No leaks, no pinholes, no setup—foolproof!



Independent-cell Cushion Uniform pressure distribution and shear reduction. Independent cells act as springs, adjusting to anatomical contours, minimizing pressure over bony prominences. Cells can also shift independently to reduce friction against skin as the user moves. Space between cells allows for air circulation. 4"H.

- #EPP1170 18"W x 16"D\$57.00 ea\$53.00 ea (3+)
- #EPP1171 16"W x 16"D66.75 ea63.00 ea (3+)

Sit longer, comfortably!

High-resilience Foam Cushions These latex-free, high-resilience foam cushions offer all-day comfort and resist bottoming out. The springy foam conforms well to the user, cradling anatomical contours. 3"H, covered in incontinence-proof, rip-stop nylon cover.

- #EPP11070 18"W x 16"D\$39.00 ea\$35.00 ea (6+)
- #EPP11069 16"W x 16"D39.00 ea35.00 ea (6+)



Incontinence-proof nylon cover



Foam cradles anatomical contours





Sit-Straight™ Cushions

Better Positioning

How Sit-Straight™ works—good support starts with a stable base




The **Wedge Cushion** shape helps to limit forward sliding in a wheelchair by keeping the hips positioned slightly lower than the knees, promoting an anterior pelvic tilt and subsequent upright sitting posture.

A traditional flat surface causes the pelvis to rest in a neutral or posterior pelvic tilt. After a short time, the user's buttocks slowly slide forward on the seat, resulting in sacral sitting. Sacral sitting puts excessive pressure on tissues that can lead to potential ulceration.

The **Solid Seat Insert** convex base design overcomes the issue of a sagging wheelchair sling seat. Most positioning or therapeutic cushions require a flat seat base for optimal function. If placed on a sagging sling seat, the cushion can buckle or tilt, ultimately negating the intended beneficial effects. The integrated convex base of these cushions fills that sling seat sag without requiring an additional seating device and without raising the height of the sitting surface.

The **Wedge Cushion and Solid Seat Insert** combination creates an immediate positive change in posture. The patient sits in a proper posture without the tendency to slide forward in his/her chair. The patient can independently participate in self-care and recreational activities. This cushion is also beneficial to caregivers as it frees them from continually repositioning patients and using restraints to keep patients from falling out of the wheelchair.

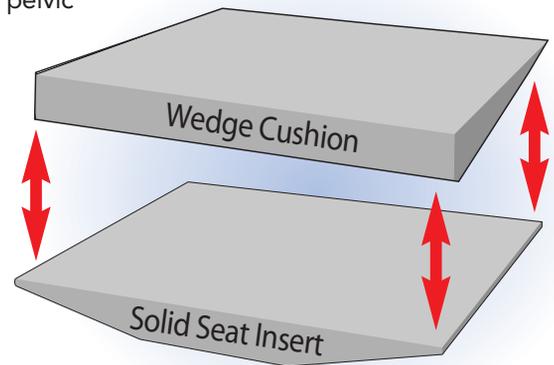

Premium firm foam cushion and base

Sit-Straight™ Basic Cushion

■ Upright posture




Combination of wedge cushion and convex base designs helps limit forward sliding and overcomes the issue of a sagging wheelchair sling seat. Increases sitting tolerance and improves wheelchair mobility by keeping the hips positioned slightly lower than the knees. Comes with wipe-clean, water-resistant vinyl cover with nonskid bottom. 16"D.

#EPP1250 18"W\$57.00 ea.....\$51.98 ea (6+)
#EPP1251 16"W\$59.75 ea.....\$54.00 ea (6+)

Sit-Straight™ with T-Gel

■ Shear reduction

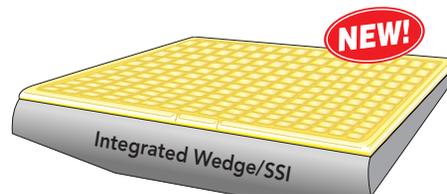


An added top layer of checkerboard T-Gel offers shear reduction and pressure redistribution. T-Gel's elastomer gel mimics fatty tissue and moves with the skin to minimize shear and skin breakdown. Tapered front edge relieves pressure on thighs. Comes with wipe-clean, water-resistant vinyl cover with nonskid bottom. 16"D.

#EPP11009 18"W\$167.75 ea.....\$161.75 (6+)
#EPP11010 16"W\$170.75 ea.....\$164.75 (6+)

Sit-Straight™ Coccyx Cushion

■ Coccyx relief

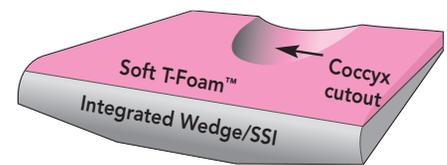


Maximum pressure relief for coccyx as top layer of soft T-Foam has a rear cutout for pressure redistribution, comfort, and conformity. Combination of wedge cushion and convex base designs helps limit forward sliding and overcomes the issue of a sagging wheelchair sling seat. Increases sitting tolerance and improves wheelchair mobility by keeping the hips positioned slightly lower than the knees. Comes with wipe-clean, water-resistant vinyl cover with nonskid bottom. 16"D.

#EPP1246 18"W ...\$109.75 ea.....\$93.75 ea (6+)
#EPP1247 16"W\$108.75 ea.....\$93.00 ea (6+)


BEST wedge cushion available

- ▶ Enhances independent functioning
- ▶ Creates a level sitting surface
- ▶ Allows longer comfortable sitting
- ▶ Limits sliding and slouching
- ▶ Reduces risk of skin breakdown

BEFORE: Poor sitting tolerance increases risk of sliding out of seat

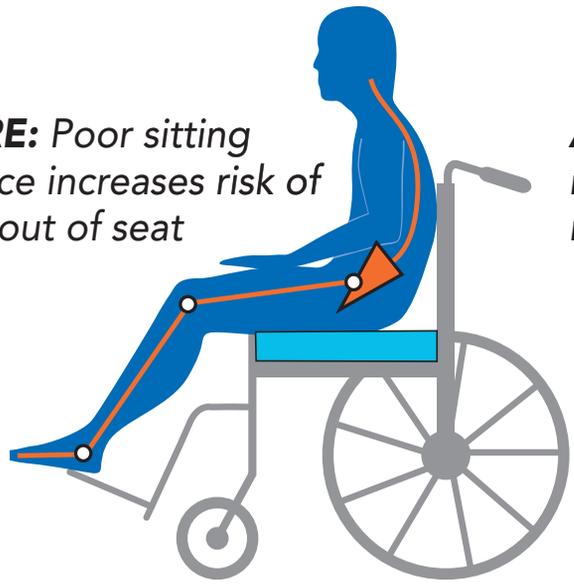


AFTER: Sit-Straight maintains hips at 90°, neutral pelvis

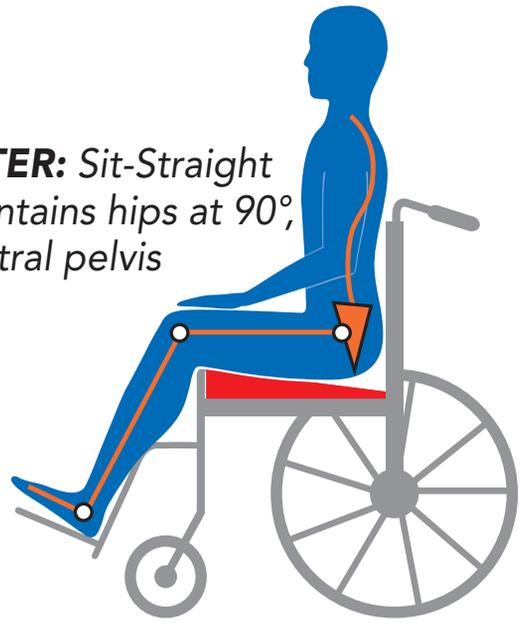


Sit-Straight™ with Semi-Rigid Pommel Cushion

Pommel positioning

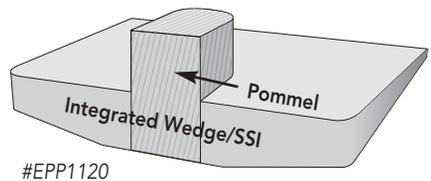

Release pommel:

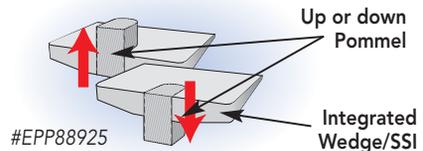

Control your resident's hip adduction and forward sliding with the built-in pommel. Great for individuals with windswept deformity or pelvic obliquity as the pommel helps keep hips and femurs aligned properly. Pommel is padded with 1" layer of soft polyfoam for added comfort. Pommel: 4"W x 3½"H. Release Pommel Sit-Straight Cushion option available to address all the needs above, but pommel drops out of the way for unhindered transfers. Comes with wipe-clean, water-resistant vinyl cover with nonskid bottom. 16"D.

#EPP1120	Semi-Rigid Pommel, 18"W.....	\$132.00 ea\$114.00 ea (6+)
#EPP1121	Semi-Rigid Pommel, 16"W.....	110.00 ea91.00 ea (6+)
#EPP88925	Release Pommel, 18"W.....	109.75 ea92.00 ea (6+)
#EPP88926	Release Pommel, 16"W.....	69.25 ea54.20 ea (6+)

Sit-Straight™ Stay-Put Coccyx Cushion

Folds with chair



Never lose a cushion again with the cushion that stays and stores on the wheelchair—even when folded. Coccyx cutout reduces pressure on the buttocks, promoting increased sitting tolerance. The Convex wedge shape fills the hammock sag and offers a flatter, more supportive positioning surface. Comes with wipe-clean, water-resistant vinyl cover with nonskid bottom. 16"D.

#EPP10955	18"W.....	\$123.75 ea	...\$107.00 ea (6+)
#EPP10954	16"W.....	123.75 ea107.75 ea (6+)

Stay-Put details, pp 46, 47.

For other Sit-Straight Cushion options, visit us at www.AliMed.com or contact us at 800-225-2610




Optimal Pressure Distribution

Why use T-Foam™?


What sets a T-Foam Cushion apart from other foam wheelchair cushions is the phenomenon called "slow recovery." When a person sitting on a T-Foam Cushion shifts his/her weight or lifts his/her body even slightly, T-Foam retains its compressed shape long enough to allow the person's capillary vessels to nourish tissues on the weight-bearing surface. The T-Foam then "recovers" its original shape.

When subjected to sudden impact, T-Foam absorbs up to 90% of impact shock and vibration. Lightweight and cool, T-Foam's open-cell structure permits air to circulate freely through the material to prevent heat buildup and perspiration. Fire-retardant, exceeds CAL 117 specifications.

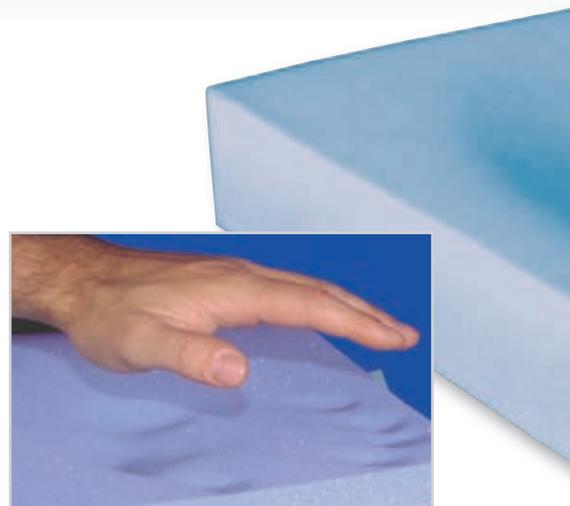

Basic T-Foam™ Cushions

Firmness ratings of Soft, Medium, and Hard should be matched to the user. In hot climates, use firmer cushions. In dry, cold environments, softer grades work better. Include black polyester knit cover. T-Foam meets flammability tests CAL 117 and FAR 25.853b.

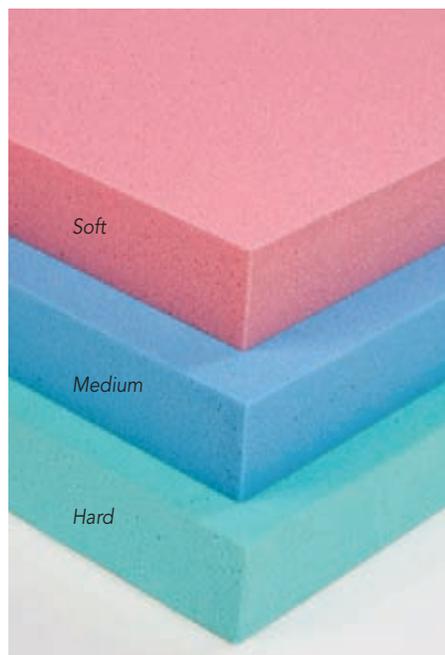

Weight Guideline Chart for T-Foam™ Cushions

MATCH THE CORRECT T-FOAM CUSHION TO THE USER



Basic T-Foam Cushions

	Soft T38	Med. T41	Hard T47	Price \$	(6+) \$
2" Thick					
18"W x 16"L	#EPP163S	#EPP163M	#EPP163H	81.25 ^{ea}	70.75 ^{ea}
16"W x 16"L	#EPP163S-JR	#EPP163M-JR	—	81.25 ^{ea}	70.75 ^{ea}
14"W x 14"L	#EPP1178	—	—	87.25 ^{ea}	65.75 ^{ea}
3" Thick					
18"W x 16"L	#EPP164S	#EPP164M	#EPP164H	99.00 ^{ea}	86.75 ^{ea}
16"W x 16"L	#EPP164S-JR	#EPP164M-JR	—	99.00 ^{ea}	86.75 ^{ea}
4" Thick					
20"W x 16"L	—	#EPP2125	#EPP2126	136.25 ^{ea}	118.00 ^{ea}
18"W x 16"L	#EPP172S	#EPP172M	#EPP172H	125.75 ^{ea}	109.75 ^{ea}



T-Foam™ delivers optimal pressure distribution

T-Foam helps reduce painful pressure points and molds to body's contours to ensure optimal pressure distribution.



Available p46

Basic T-Foam™
[Temper Foam] Cushion



Customizing T-Foam:

We can customize our T-Foam cushions for your needs! Call 800-225-2610 x103 to order.

Deluxe T-Foam™ Combo™ Cushion

■ Prevents bottoming out




Combo T-Foam fits any weight—won't bottom out on any patient up to 250 lbs.

Three 1" layers of foam. Top is conforming Soft T-Foam™. Middle is Medium T-Foam. Base is firm high-density foam, which will not bottom out. Long-term comfort and weight distribution. No weight matching required. Use on any chair with a flat seat. 3" thick. Cover is navy blue AliTex™, a two-way stretch, water-resistant fabric that "breathes" for comfort.

- #EPP1175 18"W x 16"D\$119.50 ea
- #EPP1176 16"W x 16"D119.50 ea

T-Foam™ Combo™ Foam Solid Seat Insert™ Cushion

■ Maximum pressure distribution

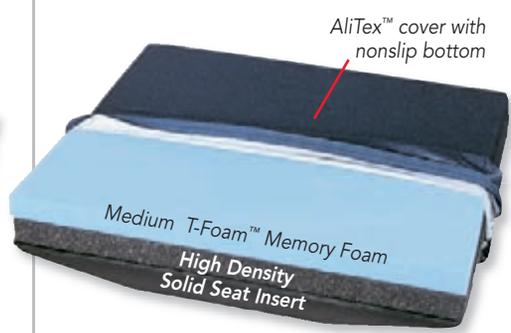

For patients up to 250 lbs. A 1" layer of Medium T-Foam over a 1" layer of high-density foam and a firm Solid Seat Insert base. 3½" thick. Black, water-resistant AliTex™ cover with grey, nonslip vinyl bottom. Zippered cover is machine-washable.

- #EPP1200 18"W x 16"L\$128.50 ea
- #EPP1201 16"W x 16"L128.50 ea

T-Foam™ Cushion with Solid Seat Insert™

■ Reduces sacral sitting



A firm convex base that fills the wheelchair sling seat and creates a flat sitting surface. Cushion combines T-Foam with a Solid Seat Insert, creating a flat sitting surface. Optimal pressure distribution with minimal change in seated height. To determine overall cushion thickness, add 1½" to the thickness of foam selected. Black, breathable polyester knit cover with non-slip vinyl. Choose between Soft or Medium foam.

- #EPP1222 18"W x 16"L x 1"D, Soft....\$102.00 ea
- #EPP1224 18"W x 16"L x 1"D, Med.....102.00 ea
- #EPP1225 16"W x 16"L x 1"D, Med.....102.00 ea
- #EPP1228 18"W x 16"L x 2"D, Soft.....110.00 ea
- #EPP1229 16"W x 16"L x 2"D, Soft.....113.00 ea
- #EPP1231 16"W x 16"L x 2"D, Med.....113.00 ea



Shear-reducing gel cushions

SPECIALTY CUSHIONS

T-Gel™

What is T-Gel™?

T-Gel is an elastomer gel—a solid sheet of soft, rubber-like polymer that mimics fatty tissue. As with typical cushions, excessive shear force leads to occlusion of blood flow, which is seen as one of the biggest contributing factors behind pressure sores and discomfort. But T-Gel moves with the skin to minimize the effects of skin-shearing and the risk of skin breakdown. It is guaranteed not to leak or dry out if accidentally punctured or cut.

Choose between two gel options:

■ Smooth

Basic shear relief. Minimizes the effects of skin shearing with smooth surface.



■ Checkerboard

Maximum shear relief. Each individual gel cell can move independently to react to the user's movements and allow greater air circulation to fragile skin.



T-Gel™ Cushion

■ Reduces shear

Excellent for applications that require minimal thickness. T-Gel polymer gel "slides" laterally, effectively reducing harmful shear forces and skin-damaging friction. Low-profile design allows use on any sitting surface. Black stretch knit cover. Smooth: 5/8" H. 5 1/2 lbs. Checkerboard: 3/4" H. 5 1/2 lbs.



Smooth

Checkerboard

Smooth

#EPP1321	16"W x 16"D\$129.00 ea.....\$105.00 ea (6+)
#EPP1319	18"W x 16"D129.00 ea.....105.00 ea (6+)
#EPP10874*	20"W x 18"D160.00 ea.....124.00 ea (6+)

*Specify Black Polyknit or Blue Rip-Stop Nylon Cover.

Checkerboard

#EPP1530	16"W x 16"D\$161.00 ea.....\$130.00 ea (6+)
#EPP1529	18"W x 16"D161.00 ea.....130.00 ea (6+)
#EPP10882	20"W x 18"D176.00 ea.....140.00 ea (6+)

T-Gel™ Plus Cushion

■ Reduces shear ■ Redistributes pressure

Combines the benefits of T-Gel with those of slow-recovery T-Foam for even better pressure distribution and shear reduction. Medium-resistance T-Foam layer magnifies the positive effects of the gel by offering extra cushion to evenly allocate coccyx pressure. Alitane® envelope protects the T-Foam from incontinence. Black stretch knit cover. Smooth: 1 5/8" H. 8 lbs. Checkerboard: 1 3/4" H. 8 lbs.



Smooth


Checkerboard

Smooth

#EPP1293	16"W x 16"D\$172.00 ea.....\$140.00 ea (6+)
#EPP1292	18"W x 16"D172.00 ea.....140.00 ea (6+)
#EPP10877	20"W x 18"D201.00 ea.....165.00 ea (6+)

Checkerboard

#EPP1532	16"W x 16"D\$182.00 ea.....\$148.00 ea (6+)
#EPP1531	18"W x 16"D182.00 ea.....148.00 ea (6+)
#EPP10885	20"W x 18"D206.00 ea.....169.00 ea (6+)

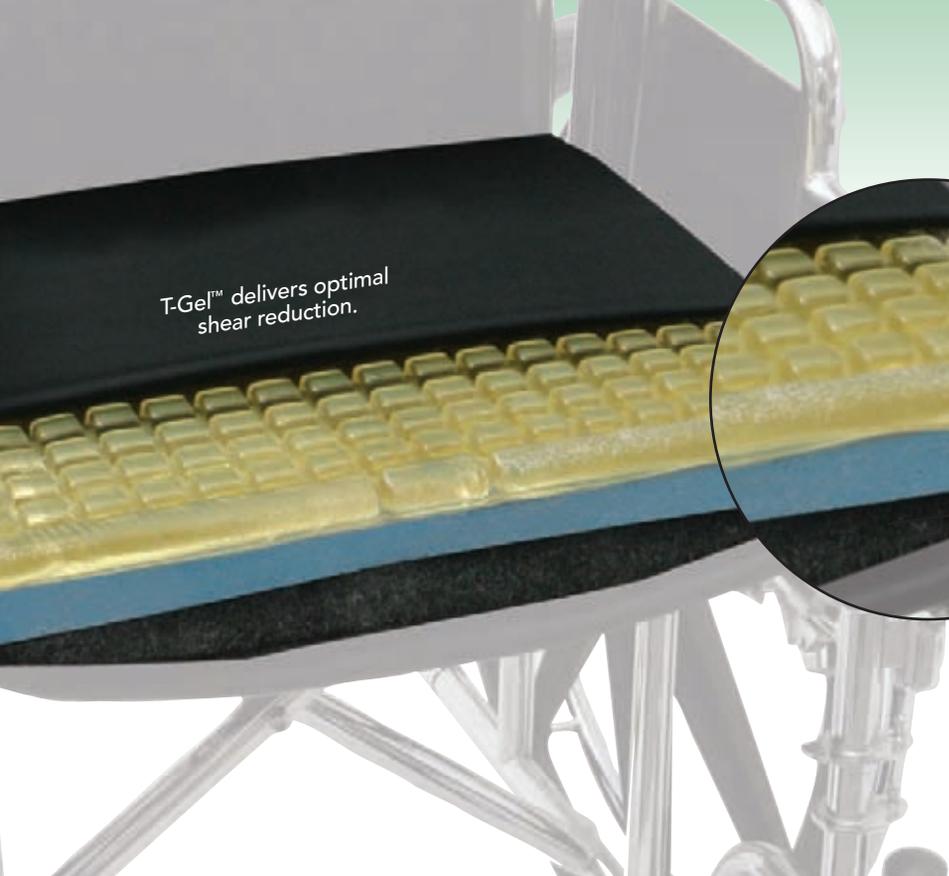

Layering Options:

T-Gel layer:
Reduces shear. Choose Smooth or Checkerboard. Details at left.

T-Foam:
Redistributes pressure.

Solid Seat Insert
Stabilizes sitting surface.



Available
p46

Customizing T-Gel:

We can customize our T-Gel cushions to meet your needs! Call 800-225-2610 x103 to order.


T-Gel™ Cushion with Solid Seat Insert™

■ Reduces shear ■ Stabilizes sitting surface

Shear-reducing gel pad sits on a convex-bottom, closed-cell-foam, solid-seat base to form a stable, flat sitting surface for sling-seat wheelchair users. Cushion reduces shear forces, while promoting upright sitting posture without increasing the overall seat cushion height. Alitane® envelope protects the T-Foam from spills and fluids. Black stretch knit cover. Smooth: 2 1/8" H. 6 lbs. Checkerboard: 2 1/4" H. 6 lbs.

Smooth

#EPP1554 16"W x 16"D\$167.00 ea\$135.00 ea (6+)
#EPP1553 18"W x 16"D167.00 ea135.00 ea (6+)

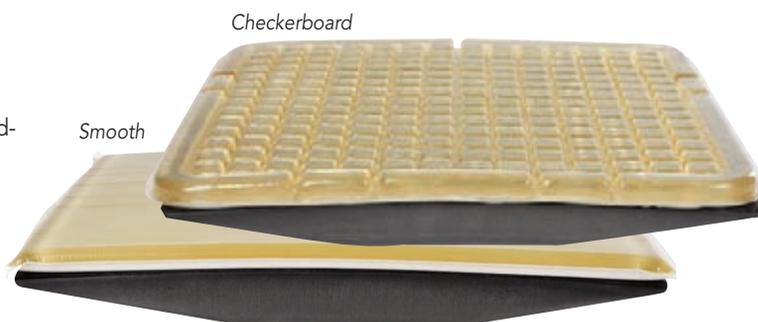

Checkerboard

#EPP1555 16"W x 16"D.....\$177.00 ea\$144.00 ea (6+)
#EPP1533 18"W x 16"D.....177.00 ea144.00 ea (6+)

T-Gel™ Plus Cushion with Solid Seat Insert™

■ Reduces shear ■ Redistributes pressure
■ Stabilizes sitting surface

Shear-reducing T-Gel pad, medium resistance T-Foam, and a convex bottom closed-cell foam solid seat base are layered together to form a cushion that addresses multiple patient needs. The cushion optimizes shear reduction, pressure redistribution and upright postural control. Black stretch knit cover. Smooth: 3 1/8" H. 8 1/2 lbs. Checkerboard: 3 1/4" H. 8 1/2 lbs.

Smooth

#EPP1295 16"W x 16"D\$193.00 ea\$156.00 ea (6+)
#EPP1294 18"W x 16"D193.00 ea156.00 ea (6+)
#EPP10880 20"W x 18"D232.00 ea196.00 ea (6+)



Checkerboard

#EPP1536 16"W x 16"D\$203.00 ea....\$164.00 ea (6+)
#EPP1535 18"W x 16"D203.00 ea.....164.00 ea (6+)
#EPP10888 20"W x 18"D243.00 ea.....207.00 ea (6+)



Gel/Foam Cushions

are intended to combine the shear-reducing properties of gel with the pressure-relieving properties of foam to create a cushion that offers comfort and skin protection.

Gel cushions have long been touted for skin protection and comfort. They minimize heat, improve sitting tolerance, and are easily wiped clean. But not all gel cushions are created equal. There are two basic types of gels used in wheelchair cushions: aqueous (water-based) and elastomer (rubber-like) gels.

Aqueous gels are viscous, must be encased in a plastic bag, and typically cost less than elastomer gels. Aqueous gel offers cooling comfort and when coupled with foam, helps to redistribute pressure away from bony prominences. Aqueous gels do not offer the same shear relief as elastomer gels and can bottom out. Water-based gels can also leak and dry out if punctured.

Elastomer gel is a solid sheet of soft, rubber-like polymer that mimics fatty tissue, moving with the user's skin to minimize the effects of skin shear. Elastomer gels won't leak or dry out if punctured (see p. 20 for our full line of T-Gel™ Polymer gel cushions).

With gel and visco foam



Contour Cushion Visco-foam pad with aqueous gel for pressure relief beneath the coccyx area. Raised front helps prevent forward thrust and sliding. Elevated sides improve lateral stability. Leg troughs promote proper thigh alignment and postural symmetry. Visco-elastic pad relieves pressure beneath the coccyx. Incontinence-proof. Anti-slip bottom. Safety strap keeps cushion safely in place. 3" H.


#EPP75 1625 16"W x 16"D\$177.00 ea
#EPP75 1630 18"W x 16"D177.00 ea
Suggested code: E2607

Gel/Foam Cushions



Gel-Foam Cushion Water-based gel lowers interface pressure and cools to eliminate perspiration. High-density foam prevents bottoming out. Low-shear outer cover reduces skin-damaging friction. Incontinence-proof vinyl cover prevents urine from entering cushion. Safety strap secures cushion to wheelchair. 2½"H.

#EPP75 1010 16"W x 16"D\$68.75 ea
#EPP75 1011 18"W x 16"D78.75 ea
Suggested code: E2601

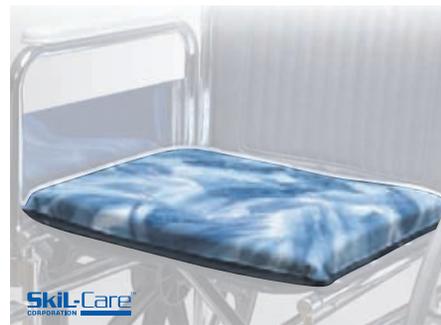

Sittin' Pretty Econo-Gel Cushions Different color surfaces serve as reminders to reposition patient. Gel-filled center pod provides pressure-relieving flotation while foam helps with comfort and stability. Incontinence-proof. Wipes clean for easy care. Safety strap keeps the wedge safely in place. 2"H.

#EPP1673 16"W x 16"D\$49.00 ea
#EPP1674 18"W x 16"D62.75 ea
Specify Bimini Pink or Bimini Blue.

Lightweight Gel Composite Cushion

Raised circles on top layer are for lightweight shear and pressure protection. 2" open-cell urethane foam with ¾" lightweight gel polymer top layer. Washable nylon cover with a nonslip base. 2¾"H.

#EPP10950 16"W x 16"D\$96.75 ea
.....76.00 ea (3+)
#EPP10951 18"W x 16"D96.75 ea
.....76.00 ea (3+)



Gel-Foam Ultra Cushion or Wedge

High-density foam increases comfort and stability while aqueous gel reduces chance of pressure ulcers. Contoured base rests below seat frame so user sits lower. Low-shear outer cover reduces skin damage. Anti-slip bottom. Safety strap keeps the cushion safely in place. 18"W x 16"D. Cushion: 4½"H. Wedge: 4½" front, slopes to 1½"H at back.

#EPP75 4330 Cushion\$125.75 ea
#EPP8782 Wedge109.00 ea



Starry Night Cushion is a two-chambered gel-foam cushion designed for maximum pressure relief. Water-based gel surface relieves interface pressure while reducing skin-damaging perspiration. Vinyl cover is incontinence-proof. 2½"H.


#EPP1670 16"W x 16"D\$77.00 ea
#EPP1817 18"W x 16"D76.75 ea
Specify size.



Gel/Foam Cushions

SPECIALTY CUSHIONS

Gel/Foam Combination

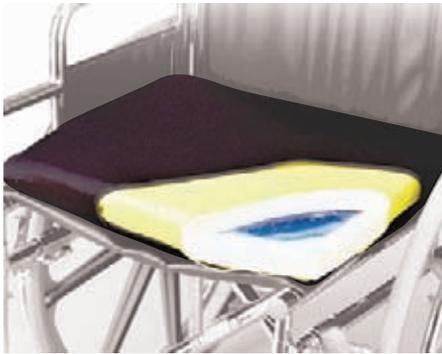

Econo-Gel Wheelchair Cushion Two-part gel pod design for greater stability and pressure-relieving flotation. Foam on top and bottom adds comfort. Two-toned covers serve as a reminder to reposition resident. Incontinence-proof. Anti-slip bottom. Safety strap keeps cushion safely in place. 2"H.

Vinyl Cover Only

- #EPP75 1154 16"W x 16"D\$36.75 ea
- #EPP75 1155 18"W x 16"D40.75 ea

Vinyl w/Polyester Cover

- #EPP8783 16"W x 16"D\$46.75 ea
- #EPP8784 18"W x 16"D50.75 ea



Multiple layers for optimal pressure reduction

Elite Cushions for moderate to high-risk users. DuraGel™ inserts for relief at critical pressure points. Moderate slope from front to back with medial/lateral thigh positioners and lateral pelvic supports for optimal posture. AT model is anti-thrust cushion with a pre-ischial bar for greater pelvic stability. Nontoxic Polymer DuraGel inserts stay in place and will not leak. Includes black, fluid-proof stretch cover. Front: 5"H at thickest point, slopes toward rear. Meets CAL 117 testing. Two-year warranty.

- #EPP1410 Elite\$159.00 ea
 - #EPP1411 Elite AT168.00 ea
- Specify 16" x 16"; 18" x 16"; 18" x 18"; 20" x 18"; 20" x 20"; 22" x 18"; 24" x 18"; 22" x 20"; or 24" x 20".
Suggested codes: E2607 and E2608



T-Gel™/T-Foam™ Combo Cushion combines the optimal shear reduction of our elastomer gel top with the pressure relief of our premium medium T-Foam for a true skin protection cushion. Raises the overall seat height by less than 2" to aid in proper positioning (also available in gel topper only or with solid seat base for added positioning support, p. 21). Incontinence-resistant cushion with washable black stretch-knit cover. Approximately 1¾"H.

- #EPP1532 16"W x 16"D\$182.00 ea
.....148.00 ea (6+)
- #EPP1531 18"W x 16"D182.00 ea
.....148.00 ea (6+)
- #EPP10885 20"W x 18"D206.00 ea
.....169.00 ea (6+)




Deluxe Gel-Foam Cushion Comfort foam encases a pressure-reducing aqueous gel bladder. Available with water-resistant removable Sure-Chek® cover or fleece cover, both launderable. Meets Cal 117 testing. 18"W x 16"D. 2"H.

- #EPP70489 Sure-Chek Cover\$97.75 ea
.....83.00 ea (2+)
- #EPP70491 Fleece Cover85.75 ea
.....73.00 ea (2+)



Convex Gel Wedge and Cushion Foam encases a pressure-reducing aqueous gel bladder. Convex bottom for level seat. Non-slip, water-repellent vinyl cover zips off. 18"W x 16"D.

- #EPP1350 Wedge, 2" to 4"H.....\$95.75 ea
- #EPP1351 Cushion, 3"H93.75 ea

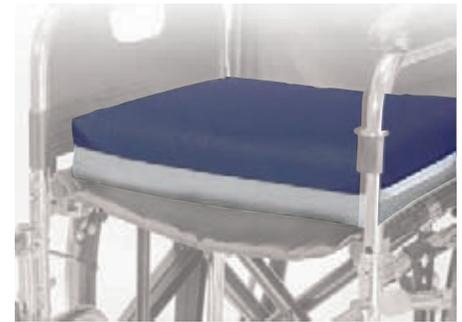

Super Economy Gel Cushion Combines pressure-reducing aqueous gel bladder surrounded by a foam cushion. Bacteriostatic and fluid-resistant, waterproof grey Weblon® bottom cover. 3"H. 12/cs. **OS**

- #EPP1272 16" x 16"\$478.00 cs (**\$39.83 ea!**)
.....466.00 cs (2+) (**\$38.83 ea!**)
 - #EPP1273 18" x 16"478.00 cs (**\$39.83 ea!**)
.....466.00 cs (2+) (**\$38.83 ea!**)
- Suggested code: E0192

Call! **VOLUME PRICING**

ROHO® Cushions

ROHO cushions feature patented Dry Flotation® air-cell technology to help reduce pressure ulcers. Each comes with a removable cover, inflator pump, and repair kit. There is no weight capacity for these cushions.

A. ROHO® Hybrid Elite™

Skin protection and stability

NEW!



Combines superb skin protection and shape-matching ability of ROHO with the stability of a Jay 3® contoured foam base. Improved pelvic alignment and stability for improved position and function.

Suggested code: K0734, K0735

B. ROHO® Harmony™

Maximum stability



Features pre-contoured foam base with interconnected air cells.

Suggested code: E2607

C. ROHO® Contour Select™

Ultimate protection/comfort



ISOFLO™ Memory Control™ with coccyx contour protection locks air in each of the four quadrants for stability by minimizing side-to-side and front-to-back motion.

Suggested codes: E0736, E0737

D. ROHO® Quadtro® Select®

Four-quadrant adjustability



Four quadrants lock air individually to achieve desired positioning. Two-way stretch cover in 2" H or 4" H air cells.

Suggested codes: K0108 (2" H), K0736, and K0737 (4" H).

E. ROHO® High Profile®

Maximum protection



Single- or dual-valve (permits adjustment to either side).

Suggested codes: K0734 or K0735 (Single), K0736 or K0737 (Dual)

F. ROHO® Low Profile®

Protection/Easier transfer



2" with interconnected air cells, single-valve operation.

Suggested codes: K0734 or K0735 (Single), K0736 or K0737 (Dual)

ROHO® Cushions		All covers include a washable stretch cover, inflator pump, and repair kit. Most cushions are available in larger sizes by special order. Call for pricing and information.												
Size	# of Cells	A. HYBRID ELITE		B. HARMONY	C. CONTOUR SELECT	D. QUADTRO SELECT		E. HIGH PROFILE			F. LOW PROFILE			
		Single Valve	Dual Valve			High Profile	Low Profile	Single Valve	Dual Valve	Repl. Cover	Single Valve	Dual Valve	Repl. Cover	
14"W x 16"D	8W x 9D	#EPP10988	#EPP10998	#EPP79355	—	—	—	—	—	—	—	—	—	—
15"W x 15"D	8W x 8D	#EPP10989	#EPP10999	—	#EPP1702	#EPP10964	#EPP10965	#EPP1130	#EPP1131	#EPP1150	#EPP1829	#EPP10971	—	
15"W x 16"D	8W x 9D	#EPP10990	#EPP11000	—	#EPP1703	#EPP10966	#EPP10967	#EPP1041	#EPP10970	#EPP1151	#EPP1042	#EPP10972	—	
16"W x 16"D	9W x 9D	#EPP10991	#EPP11001	#EPP79356	#EPP1704	#EPP1843	#EPP1332	#EPP1839	#EPP1132	#EPP1929	#EPP1126	#EPP1128	#EPP1152	
16"W x 18"D	9W x 10D	#EPP10992	#EPP11002	#EPP79357	#EPP1705	#EPP1842	#EPP1333	#EPP1838	#EPP1122	#EPP1154	#EPP1127	#EPP1129	#EPP1153	
17"W x 16"D	9W x 9D	#EPP10993	#EPP11003	#EPP79358	—	—	—	—	—	—	—	—	—	
17"W x 18"D	9W x 10D	#EPP10994	#EPP11004	#EPP79359	—	—	—	—	—	—	—	—	—	
18"W x 16"D	10W x 10D	#EPP10995	#EPP11005	#EPP79360	#EPP1706	#EPP1841	#EPP1334	#EPP1837	#EPP1832	#EPP1928	#EPP1828	#EPP1823	#EPP1923	
18"W x 18"D	10W x 10D	#EPP10996	#EPP11006	#EPP79361	#EPP1707		#EPP10968	#EPP1836	#EPP1831	#EPP1927	#EPP1827	#EPP1824	#EPP1930	
20"W x 18"D	11W x 10D	#EPP10997	#EPP11007	#EPP79362	#EPP1708	#EPP1840	#EPP10969	#EPP1835	#EPP1830	#EPP1926	#EPP1826	#EPP1825	#EPP1922	
		\$476.00 ^{ea}	\$476.00 ^{ea}	\$316.75 ^{ea}	\$461.75 ^{ea}	\$429.00 ^{ea}	\$429.00 ^{ea}	\$461.75 ^{ea}	\$481.75 ^{ea}	\$51.00 ^{ea}	\$461.75 ^{ea}	\$481.75 ^{ea}	\$51.00 ^{ea}	

Heavy Duty Covers Incontinence-resistant

#EPP81751	Low-Profile Cover*	\$71.75 ^{ea}
#EPP81752	High-Profile Cover**	71.75 ^{ea}

Specify dimensions.

*Fits (D.) Quadtro Select Low-Profile and (F.) Low-Profile models

**Fits (C.) Contour Select, (D.) Quadtro Select High-Profile, and (E.) High-Profile models

Quality alternative at a better price!

AeroCell™ II Pressure Relieving Wheelchair Cushions

The low-cost solution to helping protect patients from developing pressure sores. Air cells provide flotation and exceptional pressure redistribution. The stability of the 2" Low Cell model is great for transfers. Single-valve inflates in one step. Dual-valve permits each side of the cushion to be inflated to a different degree of firmness.

Includes bulb inflator and black stretch nylon washable cover with slip-resistant bottom • 2¼ lbs. • 440-lb. capacity • Two-year warranty

as low as
\$237.00
ea
vs. \$481.75



Air cells conform to the user's body, offering effective pressure relief.



Shown with cover.

Closeup of nonslip bottom cover

Nonslip bottom

Fishnet side panels for improved ventilation

Shown upside down

Access hole for easy valve inflation

CAPACITY CAPACITY CAPACITY
440
lbs.
CAPACITY CAPACITY CAPACITY

AeroCell II

Low Cell (2"H) Size	Single-Valve	Dual-Valve	Repl. Cover
16"W x 16"D	#EPP10727	#EPP10743	#EPP1763
16"W x 18"D	#EPP10728	#EPP10744	—
18"W x 16"D	#EPP10729	#EPP10745	#EPP1765
18"W x 18"D	#EPP10731	#EPP10746	#EPP1766
20"W x 18"D	#EPP10732	#EPP10747	#EPP1767
Price	\$295.00 ^{ea}	\$295.00 ^{ea}	\$62.00 ^{ea}
Save!	237.00!^{**} (3+)	237.00!^{**} (3+)	54.00^{ea} (3+)

Suggested code: K0736

Stays with the chair when folded!

- Helps prevent loss of cushion
- Ultimate protection/comfort

AeroCell™ Stay-Put™ Cushion The same pressure and sheer-relieving qualities you have come to trust in our standard AeroCell™ II cushion, but this version stays in place—even when the wheelchair is folded and stowed!

How does it work? The cushion and cover have a "hinged" center and buckled attachment strap (that attaches under the seat), that allows the Stay-Put Cushion to fold when the chair is folded, without damaging the cushion. When it's time to use the chair, simply unfold it as you normally would and the cushion is automatically set up and in place. Eliminates the risk of cushions being lost, placed on the wrong resident's chair, or being improperly positioned.

- #EPP10960 16"W x 16"D, Single Valve\$267.00 ea
- #EPP10961 18"W x 16"D, Single Valve267.00 ea
- #EPP10962 16"W x 16"D, Dual Valve267.00 ea

Suggested code: K0736



Patent #8,083,252B2

Sacral Relief Cushions

Excellent comfort and pressure relief



Contour Cushion

with deep cutout for sacral pressure relief.

Sculpted sitting surface for maximum pressure distribution.

Polyurethane foam. Meets CAL 117.

18"W x 16"D. Available in Firm or Soft.

Choose Flat Base or Solid Seat Insert Base.



Flat Base

Deep cutout and sculpted sitting surface available with a solid seat base at no extra cost!

Firm

#EPP1166 Flat Base\$79.00 ea\$64.00 ea (10+)



#EPP1167 SSI Base79.00 ea64.00 ea (10+)

Soft

#EPP1168 Flat Base\$79.00 ea\$64.00 ea (10+)

#EPP1169 SSI Base79.00 ea64.00 ea (10+)

Removable waterproof, nylon Navy cover

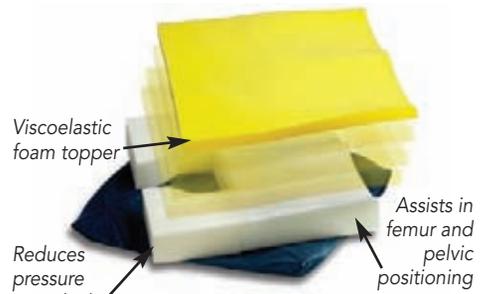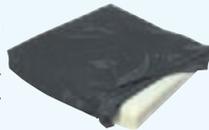

EZ-Dish Cushion®

reduces pressure on ischials and coccyx.

For moderate to high-risk users.

Contoured bottom

layer encourages healthy, upright posture and proper positioning of the legs. Comfortable viscoelastic "memory foam" top molds to the anatomically shaped, round-edged pocket below in the high-density base.

Cover has anti-slip top surface and nonskid bottom • 3"H • 18-month warranty • 250-lb. capacity




Positioning support and optional pressure relief



Super Wedge Cushion Individual cells distribute pressure better than a flat surface. Ischial cutout on the bottom further reduces sacral pressure. 18"W x 16"D.

18"W x 16"D • 2½" back edge; 4" front edge • Liquid-resistant ripstop nylon cover

#EPP1172\$72.75 ea
.....61.75 ea (10+)

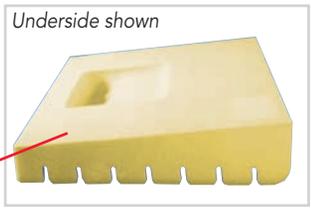

Underside ischial cutout: Foam is removed to reduce pressure!

Improves positioning!



Kyphosis cushioning—back and seat cushioning in one!

Iris® Cushion with Back

Multiple foam layers for weight distribution, durability, and pressure reduction.

Wedge cushion base is 4"H front, 3"H rear. 18-month guarantee. Waterproof polyethylene cover. No vinyl outer cover. 16"W x 32"D.

#EPP62332\$51.75 ea\$47.75 ea (3+)



Short Wave One-piece wheelchair seat and back offers relief from pain caused by sitting. Segmented fins reduce shearing and assist in proper body positioning.

Prevents sliding. Fluid-resistant, nonslip zippered cover.

#EPP71 0348 16"W\$103.00 ea

#EPP71 0349 18"W103.00 ea

#EPP71 0350 20"W117.00 ea

Suggested codes: E2601 (seat), E2611 (seatback)




SPAN America



ROHO® ENHANCER® Cushion inhibits sacral sitting, increases lateral stability, and improves leg positioning. For individuals who require additional balance support. Flexible air cells of varying heights. Includes cover, inflation pump, and repair kit.

	ea	2+
#EPP1335 15"W x 15"D	\$481.75 ea	..\$447.00 ea
#EPP1336 15"W x 16"D	481.75 ea447.00 ea
#EPP1337 16"W x 16"D	481.75 ea447.00 ea
#EPP1338 16"W x 18"D	481.75 ea447.00 ea
#EPP1339 18"W x 16"D	481.75 ea447.00 ea

Suggested code: E0736 or K0737

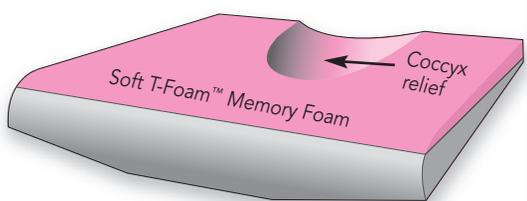


AliMed Sit-Straight™ Coccyx Cushion
 A sacral relief wedge cushion with a sculpted base—all in one cushion. Top layer of soft, pink T-Foam™ conforms anatomically. High-density foam base. Convex Solid Seat Insert™ base for a level sitting platform. Dished coccygeal area for pressure relief. Cover is breathable, waterproof, navy AliTex™ with water-resistant seams. Front height: 3"; rear height: 2".

	ea	6+
#EPP1246	18"W x 16"D...\$109.75 ea...\$93.75 ea	
#EPP1247	16"W x 16"D.....108.75 ea.....93.00 ea	

Call! **VOLUME PRICING**

Optimal skin protection

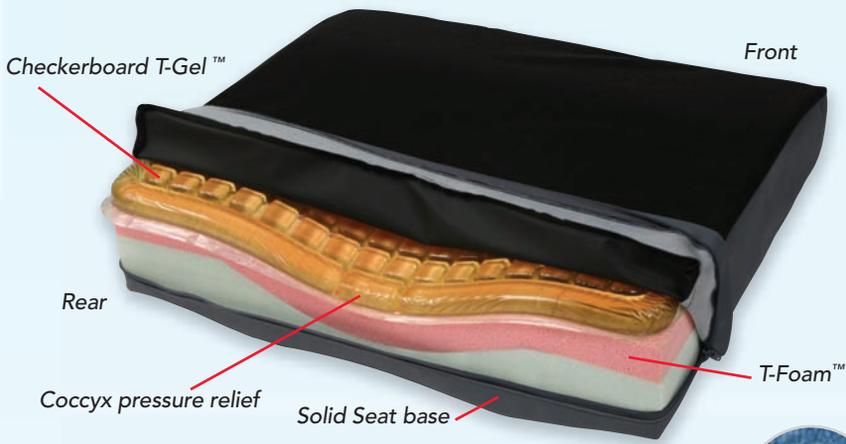

AliMed Coccyx Cushion w/Checkerboard T-Gel™ Topper
 with scooped-out area to alleviate pressure on the coccyx. T-Gel to help greatly reduce shear forces, T-Foam™ to reduce pressure, and a built-in Solid Seat Insert™ to help reduce sling seat sag, and an integrated wedge shape to help reduce forward sliding. Polyurethane-backed nylon cover with grey, nonslip bottom.



#EPP1790	Black Cover, 16"W x 16"D.....\$225.75 ea.....\$204.75 ea (3+)
#EPP1791	Black Cover, 18"W x 16"D.....226.00 ea.....205.00 ea (3+)
#EPP1792	Blue Cover, 16"W x 16"D.....237.75 ea.....216.75 ea (3+)
#EPP1793	Blue Cover, 18"W x 16"D.....237.75 ea.....216.75 ea (3+)

Stays with the chair when folded!




AliMed Sit-Straight™ with Coccyx Stay-Put™
 stays/stores on the wheelchair—even when folded. See p.46 for full details. Convex wedge shape fills the hammock sag in a fabric wheelchair seat, providing a flatter and more supportive positioning surface for the user. It also helps reduce forward sliding out of the chair. The coccyx cut out reduces pressure on the coccyx, helping increase sitting tolerance.

#EPP10954	16"W x 16"D.....\$123.75 ea
#EPP10955	18"W x 16"D.....123.75 ea

Patent #8,083,252B2

Unbeatable amputee support

AliMed Profile Sitting Orthosis 25 years of wheelchair cushion experience built in! The ischial bar and contour cuts encourage modified anterior pelvic tilt and help foster an upright sitting posture. Harmful "sacral sitting" is discouraged. Ischial pressure is relieved by contouring and shaping of the rear of the cushion. T-Foam™ interfaces between the cushion and user. Tapered distal edges relieve pressure ridge on the posterior aspect of the thigh.

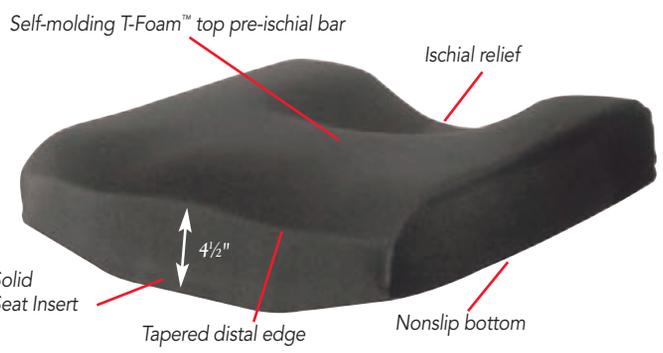

AliMed's Solid Seat Insert™ design creates a preferred sitting foundation. Cover offers breathability, incontinence protection, and a nonslip bottom. Cover can be removed to launder.

- Patient selection criteria:**
- High risk of ulcer formation and needs postural support
 - Double amputee unable to sit in a wheelchair with stable posture
 - Lower extremity amputee needing pressure protection and extra support
- | | |
|----------|-----------------------------|
| #EPP1210 | 18"W x 16"D.....\$205.50 ea |
| #EPP1211 | 16"W x 16"D.....205.50 ea |
- Suggested code: E2607

Stroke Cushions



Wheelchair adaptations for a stroke patient

For individuals who have had a stroke and have not regained enough muscle control to walk safely, a wheelchair is the means to independent mobility. However, wheelchairs were designed to be propelled with two arms, and most stroke patients have full use of only one.

Some patients can be taught to propel a wheelchair using one arm and one leg, but this only works for patients tall enough to reach the ground with their feet while seated. Here are three approaches to solve this important problem.

1. Install a "drop seat"

Not all wheelchairs are adjustable to hemi height. Drop seats replace wheelchair sling-seats to lower the seating surface by approximately 2". Drop seats install on the chair frame and require the use of a wheelchair cushion for protection. When you install a drop seat, you won't be able to fold the chair unless it is removed. This can be a major inconvenience.

Drop seat options



p36

2. Buy a Hemi-height wheelchair

A Hemi-height wheelchair has a seat that is 2" lower than a standard wheelchair. Average cost is \$400 to \$700. You'll find hemi-chairs from all of the wheelchair manufacturers we represent. Visit www.AliMed.com.

3. Use an AliMed® Stroke™ Cushion

The Stroke Cushion fills sling-seat area and protects ischial tuberosities with shear-reducing gel. Cushion nestles into concave seat without raising seat height. For added comfort, choose the Stroke Cushion with a T-Foam™ layer. See our selection of stroke cushions at right.



T-Foam Stroke Cushion



Feet can reach floor to self-propel.




Stroke Cushion w/Low-Profile Solid Seat Insert™ (SSI) Rigid closed-cell foam. Concave bottom fills a wheelchair's sagging sling seat. Cushion allows user's legs to reach the floor. Front waterfall edge reduces pressure at distal edges of thighs.

Basic Stroke Cushion with SSI™ Gel insert in the rear of the SSI offers coccyx protection. Gel/cushion interface helps prevent window edema by creating a gradual pressure transition between the firmness of the SSI and the soft, pressure-relieving gel insert.

T-Foam™ Stroke Cushion with SSI™ offers greater cushioning. Layer of 1"-thick, pressure-relieving Medium T-Foam Memory Foam lays over SSI for comfort and pressure distribution.

Pommel Stroke Cushion with SSI™ assists with hip abduction. For those experiencing a slight increase in hip adductor tone. Pommel keeps hips apart, improving alignment and minimizing tendency to slide forward. (Not available with T-Foam™ layer.)

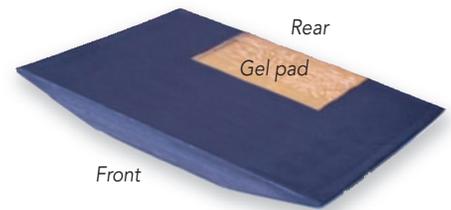

Rear

Gel pad

Front

Basic Stroke Cushion



Pommel Stroke Cushion

Stroke Cushions	18" x 16"	Price \$	16" x 16"	Price \$
Basic Stroke Cushion	#EPP1117	95.00 ^{ea}	#EPP1118	96.00 ^{ea}
T-Foam Stroke Cushion	#EPP1391	114.00 ^{ea}	#EPP1392	118.00 ^{ea}
Pommel Stroke Cushion	#EPP1545	118.75 ^{ea}	#EPP1546	124.75 ^{ea}

JAY® Duo Wheelchair Cushion Snap-in solid seat base with a contoured, soft foam cushion Fluid Pad to protect the ischial tuberosities. Maximum cushion height: 2½".

To use, remove the existing sling seat. Attach either the 7/8" - or the 1" -diameter mounting clips (both included) to the solid seat base. Snap the base onto the chair rails and attach the cushion. Two black machine-washable Quick-Dry™ covers included.



#EPP1415	16"W x 16"D, 4 lbs.	\$336.00 ea
#EPP1417	16"W x 18"D, 4½ lbs.	335.00 ea
#EPP1418	18"W x 18"D, 5 lbs.	363.00 ea
#EPP1419	20"W x 18"D, 5½ lbs.	407.00 ea

Anti-Thrust Cushions



Anti-Thrust Cushions help prevent forward sliding

These positioning cushions have a higher front surface and a lower rear surface to help prevent forward sliding in the wheelchair. The sloped region between the front and rear surfaces is positioned in front of the ischial tuberosities to keep the pelvis in a neutral position and help support upright sitting. Anti-thrust protection can be helpful when extensor tone is present. Anti-thrust cushions offer a comfortable slide-control alternative to pommel cushions, especially for male patients.


Call! **VOLUME PRICING**

Prevent forward sliding

This cushion can be
Call 800-225-2610
Customized



#EPP1974
T-Gel version

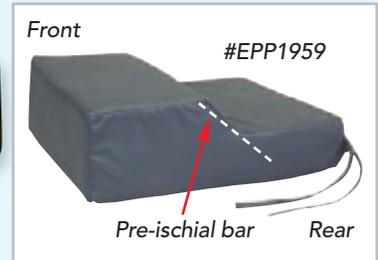

Front
#EPP1959
Pre-ischial bar
Rear



Anti-Thrust Cushions reduce forward thrust and sacral sitting while maintaining pelvic neutrality. Pre-ischial bar helps prevent forward migration of the ischial tuberosities. A great alternative to pommel cushions, particularly for men. Available in two flat base models and two convex base models. **Flat Base** models: standard flat base for solid surface seats; flat base model with solid support board to level a wheelchair sling seat. **Convex Base** models have a contoured bottom that offers a low-profile solution to the sagging wheelchair sling-seat issue by ensuring a level seating surface with minimal seat rise. Available in all foam or foam with a T-Gel overlay to reduce friction and skin shear. Flat base models have grey, nonslip vinyl cover with rear secure ties; Convex base models have a black water-resistant polyester AliTex™ top with a vinyl nonskid base.



#EPP1804
Shown w/o included cover

Standard Flat Base: 5½"H front; 3"H back • Flat Base w/Support Board: 7½"H front; 3"H back • Convex Base: 5"H front

Flat Base, 18"W x 16"D

- #EPP1959 Standard.....\$71.00 ea
- #EPP1960 w/Solid Support Board.....85.00 ea

Convex Base

- #EPP1804 Foam, 18"W x 16"D.....\$ 89.00 ea
- #EPP1931 Foam, 16"W x 16"D.....77.75 ea
- #EPP1975 T-Gel/Foam, 18"W x 16"D...138.00 ea
- #EPP1974 T-Gel/Foam, 16"W x 16"D...138.00 ea



Gel Anti-Thrust Cushion

eliminates forward sliding and sacral sitting while promoting neutral pelvic alignment. Water-based gel pod for pressure relief and cooling comfort at coccyx area. High-density foam relieves pressure. Contoured base eliminates sling-seat hammocking and minimizes seat rise. Choice of Soft or Firm base.

Fluid-proof, low-shear cover wipes clean • Rear safety straps • Anti-slip • 18"W x 16"D

- #EPP75 7115 Soft.....\$ 92.75 ea
- #EPP75 7125 Firm.....103.00 ea




Elite Anti-Thrust Cushion

combines the benefits of a contour cushion with a memory foam top layer, polymer DuraGel™ inserts beneath critical coccyx and ischial areas. Pre-ischial bar prevents forward sliding and maintains posture.

High-density foam base • Black, fluid-proof stretch cover • Meets Cal 117 • Two-year warranty

- #EPP1411\$168.00 ea

Specify 16"W x 16"D; 18"W x 16"D; 18"W x 18"D; 20"W x 18"D; 20"W x 20"D; 22"W x 18"D; 24"W x 18"D; 22"W x 20"D; or 24"W x 20"D.

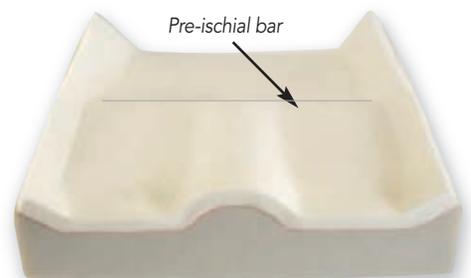

Pre-ischial bar
Pressure and shear relief—gel inserts under coccyx and ischials.

Custom and In-stock Bariatric Cushions



Custom Position Plus+™ Cushions and Wedges




are total seating systems that control sliding and lateral leaning, reduce skin shearing, and help prevent skin-damaging heat buildup. Water-based gel offers alternating pressure effect. Side pods improve lateral stability. Heavy-duty seals at sides and between pods guarantee long-term cushion integrity. Rear and front pods aid in slide control. Incontinence-proof inner cover of factory-sealed vinyl helps prevent urine from entering cushion. Two-way stretch cloth cover. 500-lb. capacity.

Important Notice: Custom Position Plus+™ Cushions and Wedges are made to the individual specifications you indicate. They are non-refundable for any reason other than manufacturing defects in workmanship or materials. If you have any questions, please call AliMed® Customer Service at 800-225-2610.

Many of our cushions
can be ordered in
larger sizes.
Call 800-225-2610
for details.



Level Cushion for Solid-Seat Wheelchair

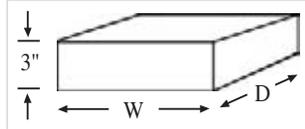


High-density foam beneath gel for stability, comfort, and pressure relief for users of solid-seat wheelchairs. 3" height helps prevent bottoming out.

- #EPP1366 18"-22"W.....\$260.00 ea
 - #EPP1373 24"-28"W.....337.00 ea
- Specify size. See chart below.



Level Cushion for Sling-Seat Wheelchair

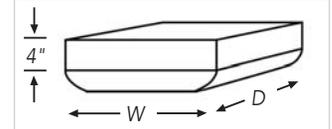


Firm, contoured foundation eliminates sling-seat hammocking, keeping pressure off the coccyx and spine and controlling internal hip rotation. High-density foam beneath the gel offers stable comfort and helps prevent bottoming out for added pressure relief.

- #EPP1368 18"-22"W.....\$260.00 ea
 - #EPP1371 24"-28"W.....337.00 ea
- Specify size. See chart below.



Wedge Cushion for Solid-Seat Wheelchair

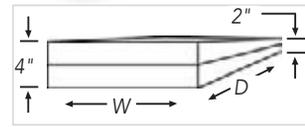


Double-layer base of firm Polylam® and high-density foam promote comfort and stability for users of solid-seat wheelchairs. Compression-resistant wedge maintains its high-front/low-back angle of elevation for slide control and positioning.

- #EPP1367 18"-22"W.....\$280.00 ea
 - #EPP1372 24"-28"W.....359.00 ea
- Specify size. See chart below.



Wedge Cushion for Sling-Seat Wheelchair

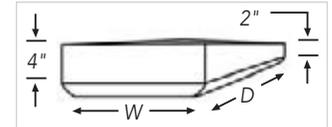

Contoured, firm foundation eliminates sling-seat hammocking to relieve pressure on coccyx and spine. Compression-resistant wedge maintains its high-front/low-back angle of elevation for slide control and positioning. High-density foam beneath the gel offers stable comfort and helps prevent bottoming out for added pressure relief.

- #EPP1369 18"-22"W.....\$280.00 ea
 - #EPP1374 24"-28"W.....359.00 ea
- Specify size. See chart below.

18"W-22"W Oversized Wheelchair Cushions and Wedges

18"W x 16"D	18"W x 18"D	18"W x 20"D	18"W x 22"D	18"W x 24"D	18"W x 26"D	18"W x 28"D
20"W x 16"D	20"W x 18"D	20"W x 20"D	20"W x 22"D	20"W x 24"D	20"W x 26"D	20"W x 28"D
22"W x 16"D	22"W x 18"D	22"W x 20"D	22"W x 22"D	22"W x 24"D	22"W x 26"D	22"W x 28"D

24"W-28"W Oversized Wheelchair Cushions and Wedges

24"W x 16"D	24"W x 18"D	24"W x 20"D	24"W x 22"D	24"W x 24"D	24"W x 26"D	24"W x 28"D
26"W x 16"D	26"W x 18"D	26"W x 20"D	26"W x 22"D	26"W x 24"D	26"W x 26"D	26"W x 28"D
28"W x 16"D	28"W x 18"D	28"W x 20"D	28"W x 22"D	28"W x 24"D	28"W x 26"D	28"W x 28"D



Shown without cover

WAFFLE® Bariatric Seat Cushion Diamond-shaped pattern to evenly distribute an individual's weight and minimize material stress. Low-air profile reduces pressure and helps prevent and treat pressure ulcers. Recommended for pain management. Four-way vinyl stretch cover is fluid-resistant, antimicrobial, and has a zippered closure. Sold separately below. 22" W x 28" D. 2" thick. 700-lb. capacity.

- #EPP10866 Cushion\$113.00 ea
- #EPP10867 Cover32.75 ea

Suggested code: E2602



Keen®

Journey™ Bariatric Cushion Tru-Relief™ Foam over high-density foam base. Incontinence-resistant, moisture-wicking LYCRA® cover.

Nonskid base • Meets CAL 117 • Available in 3" or 4" thickness • Weight guidelines: 3" cushion: 275- to 400 lb. capacity; 4" cushion: 400- to 600-lb. capacity

Cushion

- #EPP10015* 3"\$393.00 ea...\$352.00 ea (3+)
- #EPP10016** 3"436.00 ea.....396.00 ea (3+)
- #EPP10017* 4"436.00 ea.....396.00 ea (3+)
- #EPP10018** 4"477.00 ea.....436.00 ea (3+)

*Specify 22" x 18", 24" x 18", 26" x 18", or 28" x 18".

**Specify 22" x 20", 24" x 20", 26" x 20", 28" x 20", or 30" x 20".

Suggested code: E2604

30" wide or wider!



Bariatric TriFoam Cushion

Three distinct layers: A 2" top-layer of T-Foam™ memory foam for exceptional pressure distribution, a 1" middle layer of open-cell foam for pressure reduction, and a 1" bottom layer of closed-cell foam to help reduce pressure and to prevent bottoming out.




Fits wheelchairs up to 30"W • Black, waterproof cover is washable • 1,000-lb. capacity

- #EPP1795* 20"W x 18"D; 22"W x 18"D; 24"W x 18"D; 24"W x 20"D\$188.00 ea\$168.00 ea (3+)
- #EPP1796* 26"W x 18"D; 26"W x 20"D; 28"W x 18"D; 28"W x 20"D198.00 ea165.00 ea (3+)
- #EPP1797* 30"W x 18"D; 30"W x 18"D.....219.00 ea199.00 ea (3+)

*Specify size. Larger sizes available. Please call for details. Customized cushions cannot be returned.


This cushion can be
Call 800-225-2610
Customized

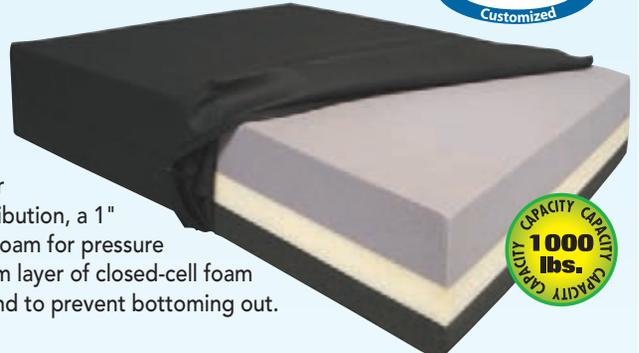

This cushion can be
Call 800-225-2610
Customized



Multiple layers for optimal pressure reduction

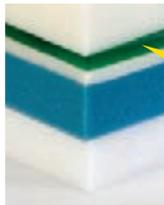

Gel

Elite Cushions for moderate to high-risk users. DuraGel™ inserts for relief at critical pressure points. Moderate slope from front to back with medial/lateral thigh positioners and lateral pelvic supports for optimal posture. AT model is anti-thrust cushion with a pre-ischial bar for greater pelvic stability. Viscoelastic high-density for long term use. Nontoxic Polymer DuraGel inserts will not leak, and stay in place. Includes black, fluid-proof stretch cover. Meets CAL 117 testing. Two-year warranty.

- #EPP1410 Elite\$159.00 ea
- #EPP1411 Elite AT168.00 ea

Specify 16" x 16"; 18" x 16"; 18" x 18"; 20" x 18"; 20" x 20"; 22" x 18"; 24" x 18"; 22" x 20"; or 24" x 20".

Suggested codes: E2607 and E2608



Pommel Cushions limit adduction... stop forward sliding

Pommel Cushions helps reduce the risk of adduction or internal rotation contracture, while limiting forward sliding in the wheelchair. They are effective for residents who require a supportive sitting surface and adduction control, who have a tendency to slide forward in the wheelchair, or for post-op hip replacement patients or those with lower-extremity spasticity. All of our pommel cushions start with a convex base, filling the sling seat to create a level sitting surface, and promoting upright posture, improved comfort, and increased sitting tolerance.

Pommel Cushion options: Standard or Wedge design, cushion material (traditional foam, T-Foam, or gel); and fixed or release pommel design.

Standard vs. Wedge: Standard models have a level cushion surface—they are appropriate for the average user and are also recommended for post-op hip patients. Wedge versions position knees slightly higher than hips to discourage forward migration of the pelvis, making them the ideal choice for “pushers” and sacral sitters with a high tendency for forward sliding.

Material options: Traditional foam is recommended for patients with no skin integrity issues; T-Foam offers excellent pressure redistribution for those with low-to-moderate skin integrity issues; T-Gel offers optimal shear reduction for moderate risk individuals.

Pommel Design: Traditionally, pommel cushions utilize a nonremovable, fixed pommel, which presents an obstacle when transferring the patient. Since the pommel prevents the user from scooting to the edge of the seat, it typically requires two caregivers to safely transfer the patient. Our revolutionary **Release Pommel** design has changed that! The pommel is independent of the cushion, attached by a secure retaining strap. When it is time to transfer, the pommel can be released (by detaching the strap) and dropped out of the way. The pommel remains tethered to the cushion by the strap, so it can't get lost! The patient can scoot forward to actively assist with the transfer, often allowing the transfer to be safely performed with only one assistant.

Pommel Cushions

Solid Seat Insert™ with Pommel An abduction wedge that also eliminates effects of wheelchair seat sag. Firm fixed pommel resists collapse for secure positioning. Surfaces that contact user are padded. For enhanced cushioning and pressure distribution, select the model with a T-Foam™ or T-Gel™ layer covering the entire sitting surface.



Built-in Solid Seat Insert is only 1½" thick at the center, fills sling seat sag. Cushion is upholstered in slip-resistant, liquid-resistant vinyl. Back ties at rear of cushion to help prevent slippage.



This cushion can be
Call 800-225-2610
Customized

	Cushion Thickness		Item #		Price \$
	Center	Edges	16"W x 16"D	18"W x 16"D	
Standard	1½"	¾"	#EPP1311	#EPP1312	82.75 ^{ea}
w/T-Foam Layer	2½"	1⅞"	#EPP1313	#EPP1314	102.75 ^{ea}
w/T-Gel Layer	2⅞"	1"	#EPP1315	#EPP1316	130.75 ^{ea}

Sit-Straight™ Wedge with Pommel reduces sacral sitting and limits forward sliding. For individuals who push and fidget or wiggle out of position. The 4"W x 3"H fixed pommel also reduces risk of adduction contracture. Padded pommel core extends to cushion base for stability.

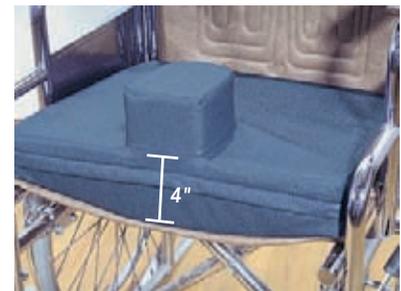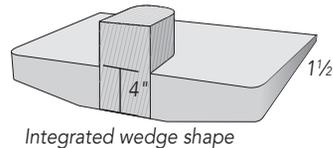

4"H in front, 1½"H in back • Dark grey, nonslip incontinence cover with front flapped zipper and ties

#EPP1120 18"W x 16"D\$132.00 ea
#EPP1121 16"W x 16"D110.00 ea

➔ **Sit-Straight™ cushions**
pp16, 17

Transfer patients with ease

Release Pommel Cushion Pommel drops out of the way, simplifying transfers. Unlike the traditional pommel cushion, this cushion allows the patient to scoot forward in preparation for the transfer. Pommel attaches with hook-and-loop strap. Solid Seat Insert™ (SSI) has a convex bottom that fills the space of the sagging sling seat to create a stable, flat sitting surface. Traditional foam. Dark grey, nonslip nylon incontinence cover.



Cushion	18"W x 16"D	16"W x 16"D	Price \$
w/SSI	#EPP88927	#EPP88926	69.25 ^{ea}
w/Wedge/SSI	#EPP88925	#EPP88924	109.75 ^{ea}

Customized cushions cannot be returned.



Amputee Wheelchair Surface

Standard version has a sturdy ½" plywood base, is padded with 1" polyfoam, and is covered with black Naugahyde®.

Premium version offers added shear relief with a thick gel layer. Attachment ties included. Both fit 18"W wheelchairs.

Bilateral surface



Universal Seat

Universal Amputee Seat allows support to be placed on the right or left side. May be extended safely to an additional 6" for a longer leg. Easily secures to wheelchair frame. 16"D seat. 10"D extension pad.

Amputee Wheelchair Surface

- #EPP70504* Standard, Polyfoam\$108.00 ea
- #EPP70505* Premium, Gel191.00 ea

Universal Amputee Seat

- #EPP1665 16"W\$376.00 ea
- #EPP1666 18"W376.00 ea

*Specify Left, Right, or Bilateral amputee.

Unbeatable support



Profile Sitting Orthosis

Pre-ischial bar and contour cuts encourage modified anterior pelvic tilt and help foster upright sitting posture. Tapered distal edges relieve pressure ridge of thigh. Solid Seat Insert™ creates preferred sitting foundation. Black breathable cover is incontinence-resistant. Nonslip grey bottom. Washable. 4½"H. 25 years of wheelchair cushion experience built in! The ischial bar and contour cuts encourage modified anterior pelvic tilt and help foster an upright sitting posture. Harmful "sacral sitting" is discouraged. Ischial pressure is relieved by contouring and shaping of the rear of the cushion. T-Foam™ interfaces between the cushion and user. Tapered distal edges relieve pressure ridge on the posterior aspect of the thigh.

AliMed's Solid Seat Insert™ design creates a preferred sitting foundation. Cover offers breathability, incontinence protection, and a nonslip bottom. Cover can be removed to launder.


Patient selection criteria:

- High risk of ulcer formation and needs postural support
- Double amputee unable to sit in a wheelchair with stable posture
- Lower extremity amputee needing pressure protection and extra support

#EPP1210 18"W x 16"D\$205.50 ea

#EPP1211 16"W x 16"D205.50 ea

Suggested code: E2607

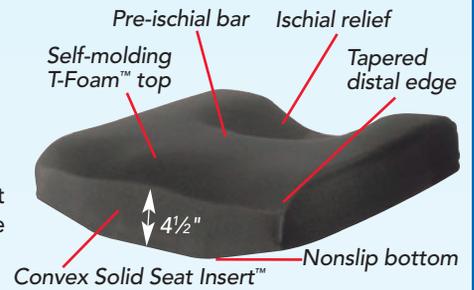

Cushion protection for Amputees.

The patient "nests" in the Profile Sitting Orthosis. Exceptionally safe, upright, stable posture results. The universal tendency to slide forward is eliminated.

T-Foam sitting surface, with Solid Seat Insert base, gives double protection from the danger of decubitus ulcer formation.



Amputee Seat Cushion Seating surface has a 6"W x 6"D extension to support the residual limb. Solid plywood base with 2"-thick firm foam covered with soft, water-resistant vinyl. Wipes clean. Safety straps secure seat to wheelchair. 2½" thick overall. Mauve.

#EPP1385 Right, 18"W x 16"D\$125.75 ea

#EPP1386 Left, 18"W x 16"D125.75 ea


#EPP1387 Double Amputee Seat Cushion, 18"W x 22"D138.00 ea

6"W x 6"D



Amputee support—add your own seat cushion

Low-Profile Amputee Seat Unpadded seat allows you to choose the appropriate cushion (not included). 11"L residual limb supports come with removable cushions with washable polyester covers. Two supports included. Main seating surface is recessed so narrow edges sit firmly on chair rails and seat rests on upholstery. Adds ¼"-½" to overall seat height.

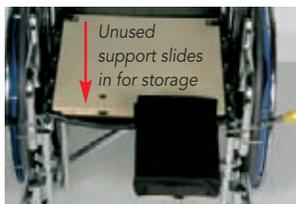

#EPP71636 16"W or 18"W\$145.00 ea135.00 ea (3+)

Specify 16"W x 16"D or 18"D; 18"W x 16"D or 18"D.

#EPP71637 20"W176.00 ea165.00 ea (3+)

Specify 20"W x 18"D or 20"D.



Unpadded Amputee Seat Cushion 2"

T-Foam™ memory foam stump support and sturdy wood seating surface. Choose your own seat cushion (not included). Use right, left, or both.

Seat base: 1½" thick • Stump support cushion surface: 6¾" W x 10"L • Stump support base: ½" thick x 23½"L

w/Cushioned Stump Support, 16"D

#EPP71000 16"W....\$273.00 ea ...\$252.00 ea (3+)

#EPP71001 18"W.....303.00 ea283.00 ea (3+)

w/o Stump Support, 16"D

#EPP71002 16"W....\$174.00 ea ...\$159.00 ea (3+)

#EPP71003 18"W.....167.00 ea152.00 ea (3+)

Cushioned Stump Support only

#EPP71004.....\$146.00 ea ...\$130.00 ea (3+)



WARNING: Extended amputee cushions can be unstable. Excess force or weight should never be allowed on the protruding, unsupported portion. Use extreme care in transfers. Cushion ties are intended to help prevent forward sliding and are not to be used as safety straps.

Sit-to-Stand Cushions

SPECIALTY CUSHIONS

Sit-to-Stand

From sit to stand without assistance



Comes with carry strap



Raises seat height



Portable Easy Up™

for post-hip surgery use or for those with arthritis of the hips and knees. Allows users to sit and stand without assistance. Adds 3" or 4" of height to any chair seat. Positions hips and knees at 90°.

Carry strap • Extra-firm foam • Zipper closure • Machine-washable fabric, navy blue • Weighs less than 1 lb. • 15"W x 15"D

#EPP1220 3"H\$58.00 ea

#EPP1221 4"H64.75 ea



Take it anywhere!

Comfort Take-Along™ Cushion Take it in the car, take it to the theater, take it anywhere you need extra cushioning for long periods of sitting. 1" of T-Foam™ for comfort and shock absorbency. 1" of high-density foam for support and resilience.

Knit cover • Carry strap • 15"W x 15"D x 2"H

#EPP1270\$72.00 ea

Lift-assist seat cushion

Uplift Seat Assist™ This comfortable easy-to-use cushion has a spring to help the user stand up. Memory foam, with water-resistant coating, to help protect against pressure sores. Waterproof cover is machine-washable in cold water. Built-in handle. Available in Standard or Plus size to accommodate users weighing between 195 and 350 lbs.

#EPP1069 Standard: 80- to 230-lb. capacity

#EPP1961 Plus: 195- 350-lb. capacity • 21 lbs. • 17"W x 19"D; 2"H closed.

#EPP1069 Standard\$180.75 ea

#EPP1961 Plus.....180.75 ea



Shop AliMed.com



- :: See our newest products
- :: Up-to-date pricing
- :: Real-time inventory
- :: Order tracking

Hip Cushions



Hip Cushions

Hip cushions have traditionally consisted of uncomfortable rectangles of firm foam with the sole purpose of raising the user up to a level that keeps the hips higher than the knees. The major issue with this style of cushion is that, while it maintains the hip at the required 90° angle, it doesn't allow the user any functional reach from the wheelchair. In order to allow the user some safe reach forward, for mobility and ADL tasks, the **AliHip™ Cushion** is gently sloped on the operative side to allow the hip to extend an additional 15° to 20°, allowing the user to lean forward the same 15° to 20° at the trunk in facilitation of sit-to-stand, self-toileting, basic ADLs, even eating!



This hip position can also help with pain control and to maximize sitting tolerance post-operatively. And because the knee naturally sits lower than the hip at this angle, individuals don't need to sit on two pillows or a cushion up to 6" high!

Comfortably maintain hip precautions

- Maintains hip at more than 90° extension
- Allows user more freedom of movement in wheelchair
- More comfortable than traditional hip cushions
- Adjusts for left or right leg



Post-Hip Surgery Cushion Patients can rise easily from any chair, including wheelchairs.

High-density foam • Wipe-clean, incontinence-proof vinyl cover • Safety ties • 18"W x 16"L x 6" H

#EPP70 7052\$44.00 ea


Post-op hip cushion



Allows operative leg to rest at an additional 15-20° extension versus traditional hip cushions

Skin safe but durable hook and loop ensure secure attachment won't irritate skin.

NEW!

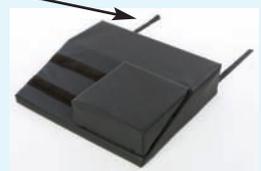

Wedge piece can be moved from left to right or removed to accommodate any hip patient.



AliHip™ Cushion is a comfortable but firm 4" foam cushion with a 15°-20° sloped cutout at the front of the sitting surface on the operative side.

How does it work? The base cushion has the 15°-20° sloped cutout at the front 2/3 of the cushion surface. The detachable wedge attaches securely on the non-operative side of the cushion, but can be removed and repositioned on the opposite side, depending on the individual patient needs. This creates a flat sitting surface for the non-operative leg, and allows the distal end of the operative leg to rest slightly lower than the hip. And the smooth but secure hook-and-loop attachments ensure proper positioning without irritating skin.

Cushion is firm yet conforming foam with choice of skin-friendly, water-resistant, black AliTex knit cover or durable black upholstered vinyl. Both cushions are removable, feature waterfall zippers and are machine washable, and feature rear safety ties. 18"W x 4"H x 16"D, slope side is 1"H in front.

#EPP11063	Knit Cover.....	\$64.00 ea.....	\$59.00 ea (3+)
#EPP11064	Vinyl Cover	64.00 ea.....	59.00 ea (3+)

NEW!

For home use



Available in left or right hip versions



Shown with optional cover

AliHip™ Cushion, Home Model This economical version of the AliHip Cushion has been designed as a take-home option for post-hip surgery patients going home with posterior dislocation precautions. This firm but comfortable foam cut-out cushion has the same sloping design on one side—the operative side. This model is left- or right-specific and is offered with an optional black polyester cover. 18"W x 4"H x 16"D, slope side is 1"H in front.

#EPP11065	Right Hip	\$30.00 ea.....	\$26.00 ea (6+)
#EPP11066	Left Hip.....	30.00 ea.....	26.00 ea (6+)
#EPP11067	Optional Cover, Right.....	19.00 ea.....	16.00 ea (6+)
#EPP11068	Optional Cover, Left.....	19.00 ea.....	16.00 ea (6+)

Drop Seats

Need a hemi-height seat but your wheelchair isn't adjustable?



Drop seats

Create a flat, low-height wheelchair seating surface without having to make adjustments to the wheelchair frame. By lowering sitting surface, you can use a higher-profile pressure-relieving cushion without sitting the patient too high in the chair, reducing fall risk. It allows user's feet to reach floor or foot supports for safer transfers, independent wheelchair mobility, and improved posture.

Most drop seats can also be set at an incline (by adjusting rear straps lower than front straps), creating a wedge effect on a traditional flat cushion. Reclined position helps limit forward sliding, reducing unwanted falls and skin breakdown.



Retrofit your older wheelchairs for hemi-height!



Drop Seat Sturdy ABS plastic attaches with strong webbing that adjusts in front and rear. Adjustable drop up to 3". Use as level seat or incline for wedge effect. Add specially configured 2"-thick AliMed Polyfoam or T-Foam™ Combo™ Cushion (1" high-density foam topped with 1" medium T-Foam) to clear crossbars. Sewn vinyl. 250-lb. capacity. Drop seat fits both 16" and 18"W x 16"D wheelchairs.



#EPP1807 Poly Cushion

Posey® Drop Seat attaches with polypropylene straps. Durable, wipe-clean polyvinyl chloride. 2" medium-density foam cushion with vinyl cover available.

AliMed


#EPP1805	Drop Seat, 18"W	\$ 85.00 ea
#EPP1807	w/18"W Poly Cushion	101.75 ea
#EPP1808	w/16"W T-Foam Combo Cushion	124.00 ea
#EPP1809	w/18"W T-Foam Combo Cushion	124.00 ea
#EPP10906	16"W Poly Cushion only	54.75 ea
#EPP10907	18"W Poly Cushion only	46.75 ea
#EPP10908	16"W T-Foam Cushion only	89.25 ea
#EPP10909	18"W T-Foam Cushion only	90.00 ea

Posey

#EPP70492	Drop Seat, 18"W	\$104.75 ea
#EPP78211	Drop Seat, 20"W	110.00 ea
#EPP70493	18"W, Cushion only	38.00 ea



Alarmed drop seat

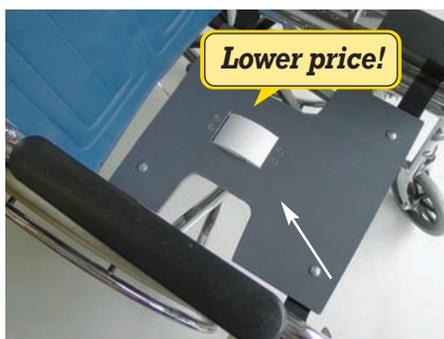

Lower price!

Drop Seat with Alarm

Adjustable drop seat with integrated 95-dB alarm that sounds when patient rises. Alarm mechanism is guaranteed for life.

Stainless steel mounting clip assembly • Fits 16"-18"W wheelchairs • 9V battery included • Adjusts to: 3", 4", or 5" below seat frame • Suspended by nylon straps.



#EPP75364	Drop Seat w/Alarm	\$168.00 ea
#EPP71088	9V Batteries, 8/pk.....	24.75 pk

Orthotic Drop Seat Contour-molded seat surface. Adjustable drop up to 3½". Create a wedge if desired. Includes removable, wipe-clean, padded vinyl cover. Fits 18"W wheelchairs.



#EPP1544.....	\$124.00 ea
---------------	-------------

JAY®

Adjustable Drop Seat Molded, reinforced, anti-slide urethane. Hook retainers secure drop seat in place. Raises or lowers in ½" increments. Snaps into place. Seat angle adjusts front-to-back or side-to-side. Use with a cushion (not included).



#EPP1135	Adult*.....	\$214.00 ea
#EPP1140	L Adult**	251.00 ea
*Adjusts 15"-18"W, 12"-18"L, 1" rise, 1½" drop. **Adjusts 18½"-22"W, 12"-22"L, 1" rise, 1½" drop.		

Adjustable J-Hook Drop Seat lowers seat up to 4". Set at level or inclined position. Two brackets keep seat from tipping forward. Lifts off for folding. Fits 16"W and 18"W chairs.



#EPP76085.....	\$167.00 ea
----------------	-------------

Stroke cushions
p28



Eliminate sling seat sag



Creates a flat sitting surface



Solid Seat Insert™ (SSI) A


sagging, curved, sling seat wheelchair is a poor sitting foundation. A flat, level surface balances the pelvis, distributes weight evenly on both ischial tuberosities, and discourages side-to-side tilt.

No more using wooden boards to bridge the sagging seat cavity raising the overall height of the sitter too high, and resulting in poor stability.

AliMed's Solid Seat Insert is a semirigid, self-cushioning foam block with a convex bottom and a flat top. The convex portion fills the concave depression formed by the sling seat. Filling the cavity is an effective way to achieve a flat sitting surface.

AliMed's shaped-foam approach offers extra cushioning, minimum seat rise, and minimal weight (8½ oz.). Seat flatness is significantly improved, limited only by the extent of original seat sag.

The SSI is smooth, waterproof, closed-cell foam. Wipes clean. No cracks or crevices to hold liquids. Choose with or without cover.

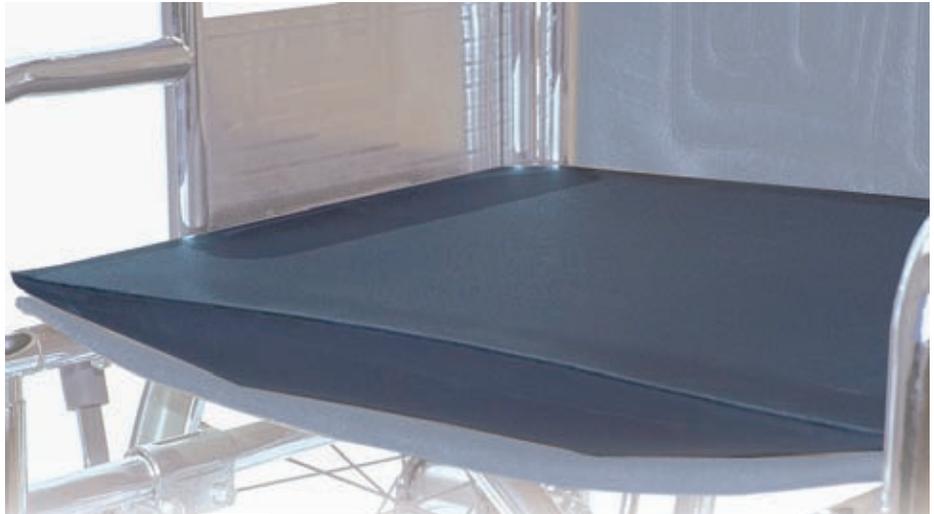

Sagging sling seat



Fills gap and makes seat level.



Ties fasten cushion to chair.



Solid Seat Insert™ (SSI) Creates a level sitting surface for any sagging wheelchair cushion.

Creates minimal seat rise so user's feet will reach floor. Evenly distributes weight and improves posture. Firm closed-end foam with a convex bottom and flat top. Wipes clean. Choose No-Slip Cover for incontinent patients or those who move and slide involuntarily.

Uncovered

#EPP1192	18" x 16"	\$45.00 ea
#EPP1193	16" x 16"	45.00 ea

w/No-Slip Cover, Grey

#EPP1260	18" x 16"	\$54.00 ea
#EPP1261	16" x 16"	54.00 ea

Decrease side leaning



Wheelchair Half Seat

Wedges Thick end (1½") fits under one side of the pelvis, raising it to a neutral position, evening out the weight-bearing area. 2/pk.



#EPP77301.....\$35.00 pk

Wheelchair Board increases overall functioning abilities. Easy to clean. Moisture resistant. Smooth, round edges. ¾" thick. Wood.

#EPP8020	18" x 18", 3½ lbs.....	\$42.75 ea
#EPP8023	18" x 16", 3 lbs.	39.00 ea
#EPP8021	16" x 16", 2¾ lbs.	35.75 ea




Call! **VOLUME PRICING**

Keen® Journey™ Cushions

Journey™ Contour and Contour Plus Positioning combines pressure relief, contoured shape, wedge design, and pommel height. Helps prevent leg abduction, forward thrusting, and sacral sliding. For moderate- to high-risk patients. Contour Plus has a higher sculpted pommel and deeper wedge design.

Incontinence-resistant, moisture-wicking covers are machine-washable, low-friction, with a rubberized, skid-resistant bottom.

See chart below to order.

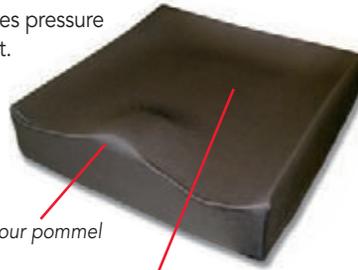

Contour pommel

Creates a comfortable wedge position.



Journey™ Elite Cushions

Tru-Relief™ foam top air base conforms and redistributes pressure horizontally. Incontinence-resistant. Machine-washable. Nonslip cover. 3" H.



Skydex base, shown without Tru-Relief™ foam top to demonstrate pressure-relieving base.



Journey™ Contour

2"	#EPP10812	\$205.00 ea	#EPP10976	\$234.75 ea	#EPP10983	\$234.75 ea
3"	#EPP10814	225.00 ea	#EPP10977	229.75 ea	#EPP10984	229.75 ea

Specify 16"W x 16"D, 16"W x 18"D, or 18"W x 16"D.

18"W x 18"D

Specify 20"W x 18"D, or 20"W x 20"D

Journey™ Contour Plus

2"	#EPP10816	\$234.00 ea	#EPP10978	\$285.75 ea	#EPP10985	\$281.00 ea
3"	#EPP10818	253.00 ea	#EPP10979	338.75 ea	#EPP10986	334.75 ea

Specify 16"W x 16"D, 16"W x 18"D, or 18"W x 16"D.

18"W x 18"D

Specify 20"W x 18"D, or 20"W x 20"D

Suggested code: Contour E2603

Suggested code: Contour Plus E2607 or E2608

Journey™ Comfort Cushions Pressure relief and comfort for long-term care residents with preventative/low-risk needs. Flat, pressure-relieving viscoelastic foam cushion conforms to any user. Available in various heights for maximum pressure relief at different weights.



2"	#EPP10803	\$ 48.75 ea	#EPP10805	\$233.00 ea
3"	#EPP10806	91.75 ea	#EPP10808	253.75 ea
4"	#EPP10809	110.00 ea	#EPP10811	233.00 ea

Specify 16"W x 16"D, 16"W x 18"D, or 18"W x 16"D.

Specify 20"W x 20"D

Suggested code: E2601 or E2602

Journey™ Comfort Convex Cushion Tru-Relief™ viscoelastic memory foam. Convex bottom fills wheelchair sling seats.

2"H cushion: 250-lb. capacity • 3" cushion: 275-lb. capacity

#EPP10019 2"H\$127.00 ea...\$112.00 ea (3+)
#EPP10020 3"H158.00 ea.....140.00 ea (3+)


Specify 16"W x 16"D or 18"W x 16"D.

Suggested code: E2601 or E2602



Journey™ Elite Contour Plus Cushion

Pressure relief for advanced wound needs. Alternative to air-cell and gel cushions. Contoured pommel top.

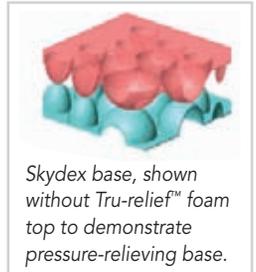


Skydex base, shown without Tru-relief™ foam top to demonstrate pressure-relieving base.

#EPP79444	16"W x 16"D\$471.00 ea
#EPP79445	16"W x 18"D471.00 ea
#EPP79446	18"W x 16"D483.00 ea
#EPP79447	18"W x 18"D483.00 ea

Suggested code: E2607

Relax™ Cushions

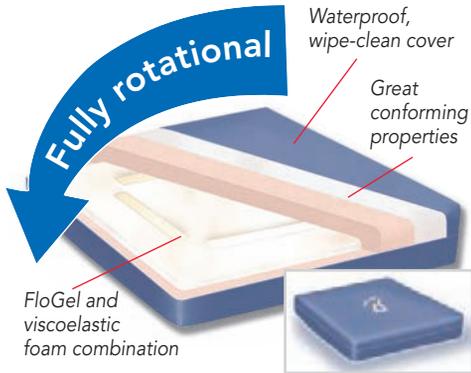

Relax™ Easy Cushion has no designated side. FloGel contained within viscoelastic foam conforms to user.

Waterproof, wipe-clean outer cover has a nonslip surface and inner incontinence-proof cover
• 280-lb. capacity* • 2½" thick

- #EPP1256 18"W x 18"D\$172.00 ea
- #EPP1257 17"W x 17"D172.00 ea




Relax™ Gelcell is a combination of air and gel. 34 air-filled cells, covered with a layer of TruGel. Air circulates to each cell, distributing weight evenly.

Stretchy, breathable cover with anti-slip base • Easy to clean • 280-lb. capacity* • 2" thick



- #EPP1989 18"W x 16"L\$218.00 ea
 - #EPP1988 17"W x 17"L218.00 ea
 - #EPP1997 16"W x 16"L239.00 ea
 - #EPP79344 18"W x 18"L203.00 ea
- Suggested code: K0734

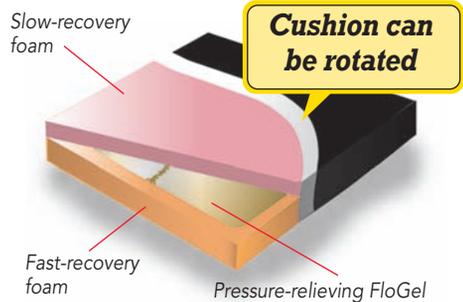

Relax™ Trio Combination of FloGel and foam conforms to the shape of the user.

Waterproof cover w/nonslip base • 180-lb. capacity* • 1½" thick

- #EPP1984 17"W x 17"D\$137.75 ea
- #EPP1985 18"W x 16"D143.00 ea
- #EPP1209 18"W x 18"D158.75 ea




Relax™ Gelcell G2 Two air chambers allow air pressure on each side of cushion to be adjusted. 48 air-filled cells covered with a layer of TruGel for shear relief.

Cells contoured towards the rear. Air circulates to each of the cells, distributing weight evenly. 280-lb. capacity*. 1½" thick.



- #EPP79346 16"W x 16"D\$223.75 ea
 - #EPP79347 16"W x 18"D223.75 ea
 - #EPP77345 18"W x 16"D223.75 ea
- Suggested code: K0734

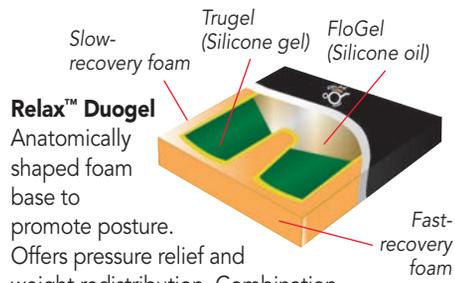

Relax™ Duogel Anatomically shaped foam base to promote posture. Offers pressure relief and weight redistribution. Combination of TruGel and FloGel.

Top/back labels ensure correct cushion placement
• Stretch, waterproof, washable cover • 280-lb. capacity* • 3" thick

- #EPP1986 17"W x 17"D\$237.75 ea
- #EPP1987 18"W x 16"D237.75 ea
- #EPP1998 16"W x 16"D237.75 ea
- #EPP1212 20"W x 18"D277.75 ea



Relax™ Ultimate Cushion Anatomically pre-contoured foam base for better positioning and posture support. Unique gel-capsule design of the silicone leg channels promotes air circulation. Clear protective polyurethane film coating is easy to wipe clean and is suitable for incontinent users. 280-lb. capacity*. 3"H.



- #EPP11040 16"W x 16"D\$190.00 ea
 - #EPP11041 18"W x 16"D190.00 ea
 - #EPP11042 18"W x 18"D190.00 ea
- Suggested code: E2607

*Cushion weight capacities assume appropriate size cushion for patient stature and chair size.

Star, Action™ and WAFFLE® cushions

StarLock™ Air Cushion

62-zone system locks each cell in place, allowing individual air cells to be isolated. 4"-high StarLock offers aggressive help for users with pressure ulcers and those at risk of skin injury.

Cushion: Flame-resistant neoprene • Antimicrobial • Washable cover: nylon and lycra top and sides with polyvinyl-treated polyester bottom; flame-resistant Doc-FF-1-70; no weight limit • Two-year limited warranty on cushion and 30-day warranty on cover • Cover, hand pump, and repair kit included • Larger sizes available by special order


#EPP10824\$411.00 ea\$381.00 ea (3+)
Choose W/D: 16" x 16"; 18" x 16"; 18" x 18"; 20" x 18"; or 22" x 18".
Suggested code: K0736



Each cell can be isolated and locked.



Low-profile

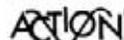

Xact® Lite Cushion Low-profile for pressure reduction and anti-shear. ¼"-thick Akton® polymer-textured overlay on a contoured multi-layered foam base promotes therapeutic positioning.

Moisture wicking stretch knit cover • Two-year warranty • 2¼" thick • 3½ lbs.

#EPP10001\$395.00 ea
Specify 16"W x 16"L, 16"W x 18"L, 18"W x 16"L, or 18"W x 18"L.
#EPP10002\$395.00 ea\$334.00 ea (3+)
Specify 20"W x 16"L, 16"W x 20"L, 20"W x 18"L, 18"W x 20"L, or 20"W x 20"L.
Suggested code: E2603



Shown without cover

WAFFLE® Original Seat Cushion

Features a unique low-profile design to provide safety and convenience for getting in and out of chairs. Air venting holes allow heat and moisture to easily dissipate for higher level of comfort. Utilizes static air technology in the prevention and treatment of pressure ulcers up through Stage IV. 17"W x 17"D. 1.5"H. 300-lb. capacity.

#EPP10860 Cushion\$42.75 ea
#EPP10867 Cover\$32.75 ea



Shown without cover

WAFFLE® Bariatric Seat Cushion

Diamond-shaped pattern evenly distributes an individual's weight and minimizes stress on cushion. Low-air profile reduces pressure and helps prevent and treat pressure ulcers. Recommended for pain management. Four-way vinyl stretch cover is fluid-resistant, antimicrobial, and has a zippered closure. Sold separately below. 22"W x 28"D x 2" thick. 700-lb. capacity.

#EPP10866 Cushion\$113.00 ea
#EPP10867 Cover\$32.75 ea
Suggested code: E2603



Contour cushion



Xact® Classic Cushion for mid- to high-risk users. Reduces and distributes pressure to low-risk areas while maintaining proper skin temperature. Gel cube overlay on contoured viscofoam pad with sealed solid foam base. High laterals, moderate pommel, and a pre-ischial ridge help maintain postural support and a neutral pelvis.

Moisture-wicking stretch-knit cover • Machine-wash/-dry • Two-year warranty • 4¾" thick • 6.8 lbs. • 250-lb. capacity

#EPP10003\$395.00 ea\$344.00 ea (3+)
Specify 14" x 14", 14" x 16", 16" x 16", 16" x 18", 18" x 16", or 18" x 18".
#EPP10004\$460.00 ea\$429.00 ea (3+)
Specify 16" x 20", 18" x 20", 20" x 16", 20" x 18", or 20" x 20".
Suggested code: E2607 Skin Protection and Positioning

Shop **AliMed.com**

- :: See our newest products
- :: Up-to-date pricing
- :: Real-time inventory
- :: Order tracking

JAY® Cushions

SPECIALTY CUSHIONS



JAY® Duo Wheelchair Cushion Positions user closer to the floor for foot propulsion. 2½" deep. Solid seat base with a contoured, soft foam cushion. Fluid Pad to protect ischial tuberosities. Includes mounting clips and two machine-washable black covers.

- #EPP1415 16"W x 16"D, 4 lbs.\$336.00 ea
 - #EPP1417 16"W x 18"D, 4½ lbs.....335.00 ea
 - #EPP1418 18"W x 18"D, 5 lbs.363.00 ea
 - #EPP1419 20"W x 18"D, 5½ lbs.....407.00 ea
- Suggested code: E2603

JAY® Basic Mildly contoured foam cushion. Beveled base for level sitting surface.
Liquid-resistant black cover w/nonskid bottom
• One-year warranty • 2" thick

- #EPP1194 16"W x 16"D\$148.00 ea
- #EPP1195 18"W x 16"D148.00 ea
- #EPP1196 16"W x 18"D148.00 ea
- #EPP1197 18"W x 18"D148.00 ea
- #EPP1198 20"W x 18"D148.00 ea

Suggested code: E2601 or E2602



JAY® J2 Cushion combines the oversized, trisection Jay Flow™ Fluid Pad with a lightweight, closed-cell (molded) foam base. Pad reduces shear and offers contoured seating surface and optimal gel protection.

Fluid Pad attaches to base with color-coded hook-and-loop straps. Black, ballistic stretch cover with nonskid bottom included. For use on a flat-base wheelchair with a sling-seat chair using a J2 Solid Seat Insert, available separately below.

Suggested code: K0734



J2 Solid Seat Insert (not shown) fits into the sling seat to create a flat seating surface. Honey-combed aluminum covered in black vinyl. Not recommended for use with other cushions.

Jay Flow™ Fluid Pad



Base



Slip-on Cover



Don't see what you're looking for?
We can order it for you.

ROHO®
JAY®
Skil-Care™
Keen® Mobility
Action®
Star
Posey®

Any model,
any size.

Call us!
800.225.2610
x103

		J2 Cushion		J2 SSI	
W x D	Weight	Item #	Price \$	Item #	Price \$
18" x 16"	8 ¾ lbs.	#EPPJY 2100	383.00 ^{ea}	#EPPS2100	52.75 ^{ea}
18" x 18"	8 ¾ lbs.	#EPPJY 2101	375.00 ^{ea}	#EPPS2101	67.75 ^{ea}
18" x 20"	10 ¾ lbs.	#EPPJY 2103	414.00 ^{ea}	#EPPS2103	67.00 ^{ea}
16" x 16"	7 ½ lbs.	#EPPJY 2105	375.00 ^{ea}	#EPPS2105	67.75 ^{ea}
14" x 16"	6 ½ lbs.	#EPPJY 2108	387.00 ^{ea}	#EPPS2108	66.75 ^{ea}

JAY® Care Cushion Lightweight, one-piece cushion. Contoured urethane foam base with Jay Flow™ Fluid Pad. Gel pockets protect ischial tuberosities and create even distribution of gel under legs. For use on sling-seat upholstery. Do not use with a solid seat insert that is placed on the seat rails or in a chair with overslung upholstery. Black ballistic stretch cover with nonskid bottom. ¾"H.

- #EPPJY3100 18"W x 16"D, 9 lbs.\$327.00 ea
 - #EPPJY3105 16"W x 18"D, 8½ lbs.....345.00 ea
- Suggested code: E2607



Call! **VOLUME PRICING**

Fall Prevention Cushions with Built-in Alarms



Reducing falls

Appropriate wheelchair seating is an important factor when considering a comprehensive approach to fall prevention. Over time, an individual sitting in a wheelchair is affected by different factors—muscle fatigue, balance deficits, pressure on soft tissue—all of which can cause shifting or scooting on the seating surface. The end result is often a dangerous fall off the front edge of the wheelchair cushion. This doesn't have to happen! AliMed offers a number of different cushion options to help reduce the risk of forward sliding and wheelchair falls.

Cushions with Alarms One strategy is to use a sensor type alarm to alert staff when a patient begins sliding forward on the seat. When pressure is removed from the sitting surface of the cushion, the alarm unit sounds, letting the caregiver know that the patient needs help to reposition.

Positioning Cushions Another strategy is to use a cushion that helps limit forward sliding in the first place. There are a number of cushions that do this, and all share the concept of a sloping design where the front of the cushion is higher than the back.

Positioning Cushions. Details, p.44

No-slip sensor pad and cushion in one

 **Sensor Cushion Alarm** is a comfortable foam wheelchair cushion that doubles as a sensor pad alarm. But unlike the traditional separate cushion and alarm/pad unit, this unit always stays in place, eliminating sensor pad slippage. Pressure-sensitive pad is embedded in 3"-thick fire retardant, cushioning foam.


Alarm alerts caregiver when patient begins to get out of wheelchair, helping prevent falls. Cushion fits both 16" and 18"W chairs. Upholstered in black, waterproof vinyl with two ties for attachment. Includes cushion, alarm unit, 9V battery, universal mounting bracket, and an AliTie™ to keep the alarm cord bundled.

Cushion: 17"W x 16"D x 3"H • **Alarm:** On/off switch
• 85-dB or 100-dB setting • Low-battery warning • Chime tone
• Mounting bracket

#EPP75049 Sensor Cushion Alarm.....\$119.00 ea
#EPP75448 Repl. Sensor Cushions.....73.00 ea
#EPP77141 Repl. Basic Sensor Alarm Unit.....50.00 ea



#EPP75049

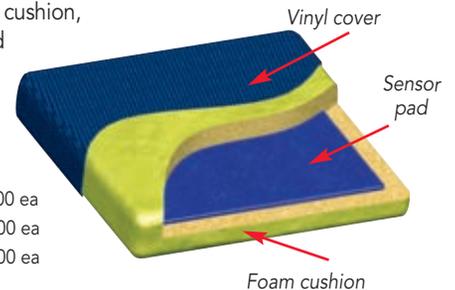

Foam cushion



View our full line of alarms in our
**FREE comprehensive
Patient and Staff Safety catalog**

Call 800-225-2610
Item #EPP1305



MAIL AliMed® inc.

Order Department, 297 High St., Dedham, MA 02026

- PHONE 800-225-2610 8:30 AM–6:00 PM EST, Mon.–Fri.
- 24-HOUR FAX 800-437-2966
- International: 781-329-2900 FAX 781-329-8392
- ONLINE CATALOG 24-hr. web ordering www.AliMed.com

CHARGE IT



By phone, mail, or fax. Please see our new Shipping Rate Table at right for exact charges. Credit cards are only accepted at time of order entry.

- QUESTION OR RETURN? Customer Service, x6901

MINIMUM BILLING Orders are subject to a \$25.00 minimum merchandise billing plus shipping, handling, and insurance. International orders (except Canada) require a \$300.00 (U.S.) minimum order.

TERMS Net 15 to established accounts. All prices F.O.B. shipping point. Shipment made at prevailing price at time of order. All prices in U.S. \$. Prices are professional wholesale. Patient may order directly; retail pricing applies. Prices, availability, and specs subject to change. Visit www.AliMed.com for the most current pricing. Items may be discontinued at any time. AliMed reserves the right to substitute similar products to those shown in catalogs at any time. Overdue accounts are subject to a late payment fee of 2.0% per month, resulting in an annual rate of 24%.

Volume Discounts: Item numbers, sizes, colors, Lefts and Rights, etc. cannot be assorted for a quantity discount. One item #, one style per volume discount order (unless otherwise specified).

SHIPPING We ship FedEx/UPS whenever possible. Charges are prepaid and added to invoice. Freight shipments are freight collect. See Shipping Rate Table for exact charges.

 Oversized items subject to additional freight charges.

OVERSIZED CHARGES Apply to all orders including contract orders, items with free ground shipping, and free shipping promotions.

GLN #073365700009

TAX Applies to all orders shipped in MA, MI, SC, SD, and WA. If you are tax exempt, please forward a copy of your tax exempt or resale certificate. If your facility is not in MA, MI, SC, SD, or WA, please submit use tax directly to your state.

INSURANCE Your shipment is 100% insured against loss or damage.

RETURNS Our return policy is consistent in maintaining the best value and highest service level for all of our customers. If the merchandise you received is defective or not as you ordered, please contact our customer service staff for quick resolution.

All other returns will be subject to a 20% restocking fee (\$10.00 minimum) except "Special Order" items that have been "drop shipped". (These items will be subject to a restocking fee that may vary at the discretion of the individual vendor.)

The guidelines below must be followed in order to receive any credit:

- Merchandise credit only
- Product must be in new, resaleable, unmarked condition and in original packaging
- Merchandise must be returned within 30 days of invoice date
- Custom products, beds, books, videos, shoes, eating utensils, and toileting products are not returnable
- Open packages cannot be returned for credit



To ensure prompt return processing, please attach the original packing list, invoice or RA number to the return. Refunds will be for merchandise value only. Please allow three to four weeks for return processing.

Double protection! Controls forward sliding too!

AliMed Sit-Straight™ Wedge Cushion Alarm System combines the positioning benefits of our Sit-Straight™ Cushion with fall prevention benefits of an integrated sensor pad alarm system for double the protection. The wedge design and flat sitting surface encourage upright sitting and decrease the tendency to slide forward on the seat surface. But for added protection, a sensor pad is imbedded within the foam of the cushion to detect when weight is lifted off the seat surface. And because the pad is embedded, there is no need to worry about a sensor pad slipping off the cushion.

Select the system with our tamper-resistant TR2 Alarm for added protection from patient tampering or staff/caregiver error (see below for details), or choose the economical Standard alarm unit option.

Cushion: 3½" H front • 1½" H rear • Wipe clean • Waterproof, nonslip cover with rear ties • Black
Alarm: 85- or 100-dB alarm settings • Chime or Alarm alert options • Two concealed reset buttons • Low-battery light • Nurse call jack • Rubber bumpers • For use with sensor devices only—sensor pads bed/chair, seatbelts, bed belt, cushions • 4 AA batteries, included

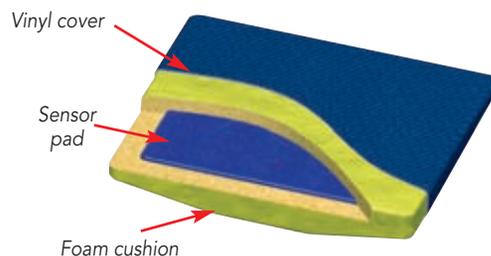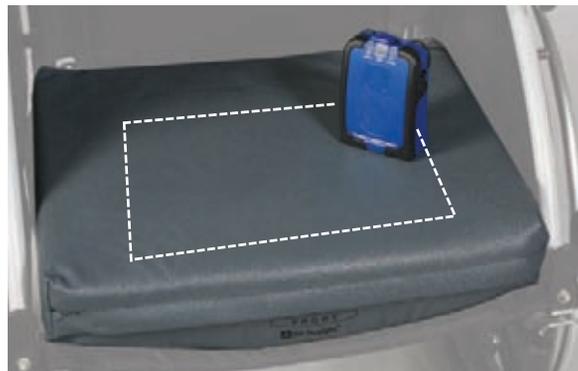

Sit-Straight Cushion with TR2 Alarm System

Item #	Size	Each \$	Buy 6 or More Cases
#EPP70151	18"W x 16"D	\$148.75 each	\$133.75 each
#EPP70152	16"W x 16"D	151.75 each	136.75 each

Sit-Straight Standard Alarm System

Item #	Size	Each \$	Buy 6 or More Cases
#EPP75378	18"W x 16"D	\$115.75 each	\$98.00 each
#EPP75379	16"W x 16"D	118.75 each	100.75 each

Replacement Alarm Units

#EPP710985	TR2	\$84.00 each
#EPP77141	Standard	50.00 each

AliMed TR2 Tamper-Resistant Alarm Online video demonstrations!

 <http://video.AliMed.info>
OR get a FREE DVD. #EPP71 1973



TRUCK SHIPMENTS Delivery Inspection Required

IMPORTANT NOTICE: All truck shipments must be fully inspected by receiver before signing for the delivery to ensure there is no concealed damage. If you cannot inspect immediately, make a note "pending inspection" with your signature. Any damaged shipment received should be refused by the receiver. These guidelines must be strictly followed in order to receive any credit on Truck shipment orders. **Overnight Truck Shipments – AliMed does NOT provide overnight delivery on orders that require delivery by truck.**

DAMAGED GOODS/SHORT SHIPMENTS Must be reported within 10 days of receipt. Be sure to note any damage or poor box conditions when signing for packages. If shipment arrives damaged or with parts missing, contact the carrier immediately. Be sure to save product packaging. All shipping charges and risk of loss are FOB shipping point and are the responsibility of the purchaser.

Shipping Rate Table*

Includes shipping and handling charges • Temporary fuel surcharge may be added to our published rates • No small order charge • Instant freight quotations • No additional freight or backorders
Some large volume orders and/or oversized items may require additional freight charges or truck shipping.

Ground/Air (FedEx, UPS, Postal) Shipping, handling and insurance

Merchandise value	Inside Continental US	Outside continental US** and Canada
\$ 00-99.99	\$11.99	\$26.99
100-199.99	16.99	35.99
200-499.99	19.99	42.99
500-799.99	28.99	49.99
800+	34.99	55.99

COD orders not available. For 2nd Day, Next Day Air, or Saturday Delivery, additional charges apply to above rates. Refer to www.AliMed.com for details. Normally, orders are shipped within 24 hours. For same-day guarantee rush shipment, add a \$15 special handling charge. For hazardous products, add \$25 for each individual item to above rates. Hazardous items must ship ground. Collect shipments are subject to a \$7.50 processing fee per order and can only be processed when items are shipped direct from AliMed's warehouse. Any items shipped from an alternate site will have charges prepaid and added to the invoice as per our shipping table.

Truck Shipments (In addition to any applicable Ground Shipments.)

Merchandise value	Shipping and handling Continental US only
\$ 00-299.99	\$145.00
300-499.99	195.00
500-999.99	225.00
1000-1999.99	280.00
2000-3599.99	380.00
3600+	10% of order value

No COD orders available. Products are delivered to your receiving dock. Inside delivery and liftgates are additional. International shipping prices, including Canada, reflect the base shipping charge, not including possible customs duties, taxes, and other country specific fees.

* Shipping rate table does not apply to International shipments. International shipments (except Canada) require a freight quote. Contact our International department for assistance.
 ** Shipments to HI, AK, and Puerto Rico are sent by 2nd Day Air, unless otherwise requested. Shipments to Canada may incur additional charges, including without limitation, duties, taxes or country specific items.

Fall Prevention Cushions

Reduce forward sliding...help reduce falls



Cushions designed to prevent forward sliding

Cushions can be used as alternatives to belts or restraints to help prevent forward sliding in a wheelchair, which can easily result in a fall. A cushion design that is higher in the front and lower in the back uses natural body mechanics to do so by placing the knees slightly higher than the hips. This position naturally tips the pelvis slightly forward, decreasing the tendency to slouch and slide forward on the seat cushion. There are three basic cushion designs that achieve this: A basic wedge, a pommel, and an anti-thrust.

Wedge Cushions come in all shapes and configurations, but the basic principle is the same—the front-to-back slope encourages an anterior pelvic tilt. Wedge cushions also lend themselves well to modifications that assist in skin protection. Options include: coccyx cutouts, gel toppers, and contoured cushion surfaces.

Pommel Cushions work differently. They use a center pommel at the front of the cushion that physically blocks the user from sliding forward. Pommel cushions have the added benefit of positioning the hips in slight abduction for patients at risk for developing hip adduction contracture secondary to fracture or abnormal tone issues. Our Sit-Straight™ Wedge with Pommel combines the wedge and pommel designs for added benefit.

Anti-Thrust Cushions use a similar design concept to a wedge, but are more effective than a straight wedge at reducing forward sliding. They utilize a quick slope in height from a high flat surface in the front to a lower flat surface in the rear. The sloped region (known as the pre-ischial ridge) is placed anterior to the ischial tuberosities to keep the pelvis in neutral and prevent forward migration. Anti-Thrust cushions are ideal for patients who actively push their hips forward on the seat, and are a good alternative to pommel cushions for many men and heavier women.

Other considerations

When choosing any of these cushions, consider a flat or convex (solid seat insert) base, the cushion material (foam type and density, or gel), and cover material. If you are fitting the cushion for a sling seat wheelchair, a convex base and nonslip cover fabric with rear safety ties are always a good bet. All of our wedge, pommel, and anti-thrust cushions come in flat or convex base models and most have rear safety ties.

WEDGE



Basic Wedge Cushion This simple wedge design creates a comfortable sitting surface that discourages slouching and maximizes sitting tolerance. High-density foam contours to body for added support, yet holds its shape even with long-term use. Cushion tapers from 3" H in front) to 1" H in rear.



High-density foam cushions, supports.

Cover options: Choose between two removable covers. The **Nonslip Vinyl Cover** is a wipe-clean, nonslip vinyl fabric that further limits forward sliding and is great for long-term durability. **Knit Cover** is a washable polyester knit fabric that further enhances comfort and reduces heat buildup.

Nonslip Vinyl Covers

#EPP260	18"W x 16"D.....	\$46.00 ea.....	\$38.75 ea (10+)
#EPP261	16"W x 16"D.....	46.75 ea.....	38.75 ea (10+)

Knit Covers

#EPP1088	18"W x 16"D.....	\$46.00 ea.....	\$39.97 ea (10+)
#EPP1089	16"W x 16"D.....	64.75 ea.....	57.75 ea (10+)




Sit-Straight™ Coccyx Wedge Cushion Combines the classic wedge design with a sacral relief cutout, a convex base, and a top layer of comfortable, pressure-relieving soft T-Foam for maximum skin protection as well as reduction in forward sliding. Cover is breathable, waterproof, AliTex™ with water-resistant seams. 3" H front, 2" H rear.


#EPP1246	18"W x 16"D.....	\$109.75 ea.....	\$93.75 ea (6+)
#EPP1247	16"W x 16"D.....	108.75 ea.....	93.00 ea (6+)

[More Wedge Cushions p13](#)

POMMEL

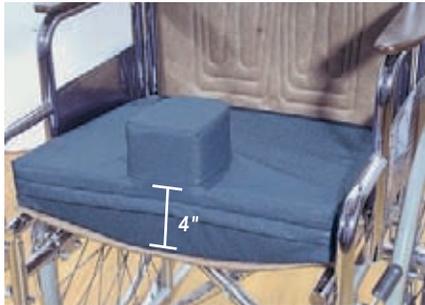

AliMed® Sit-Straight™ Wedge with Pommel reduces sacral sitting and limits forward sliding. For individuals who fidget or wiggle out of position. The 4"W x 3"H pommel also reduces risk of adduction contracture. Padded pommel core extends to cushion base for stability.

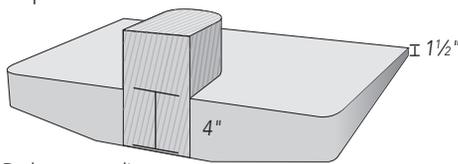

4" H in front, 1½" H in back • Dark grey nonslip incontinence cover • Front flapped zipper and ties

- #EPP1120 18"W x 16"D\$132.00 ea
- #EPP1121 16"W x 16"D110.00 ea



AliMed® Solid Seat Insert™ with Pommel An abduction wedge that also eliminates effects of wheelchair seat sag. Firm pommel resists collapse for secure positioning. Surfaces that contact user are padded. For enhanced cushioning and pressure distribution, select the model with a T-Foam™ or T-Gel™ layer covering the entire sitting surface.

Built-in Solid Seat Insert is only 1½" thick at the center, fills sling seat sag. Cushion is upholstered in slip-resistant, liquid-resistant vinyl. Back ties at rear of cushion to help prevent slippage.

	Cushion Thickness		Dimensions		Price \$
	Center	Edges	16"W x 16"D	18"W x 16"D	
Standard	1½"	¾"	#EPP1311	#EPP1312	82.75 ^{ea}
w/T-Foam Layer	2½"	1¾"	#EPP1313	#EPP1314	98.25 ^{ea}
w/T-Gel Layer	2½"	1"	#EPP1315	#EPP1316	126.75 ^{ea}

More Pommel Cushions p32


ANTI-THRUST



AliMed® Anti-Thrust Cushion reduces forward thrust and sacral sitting while maintaining pelvic neutrality. Recommended for men as an alternative to the Pommel Cushion. Available with Solid Support Board (not convex) to fill wheelchair sling seat.

Anti-Thrust Cushion: 16"W x 18"D x 5½"H front, 3"H back
Cushion w/Solid Support Board: 16"W x 18"D x 7½"H front, 3"H back
 • Secures with safety ties • Grey nonslip cover

- #EPP1959 Cushion.....\$71.00 ea
- #EPP1960 w/Solid Support Board85.00 ea

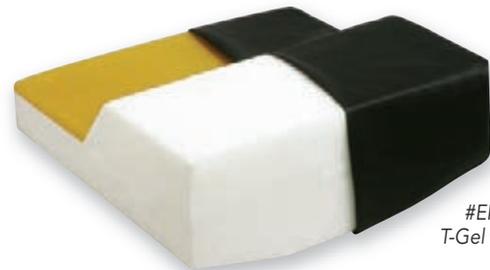

#EPP1974
T-Gel version

AliMed® Low-Profile Anti-Thrust Cushion Foam cushion with integrated convex Solid Seat Insert™ base shaped to ensure level seating with minimal seat rise.

5"H at front edge • Covered in AliTex™, a polyester knit outer and polyurethane, water-resistant inner surface fabric
 • Nonslip vinyl base • Available with T-Gel™ Overlay for reduction of shear force on skin.



#EPP1804
Shown without included cover

w/o T-Gel Overlay

- #EPP1804 18"W x 16"D\$ 89.00 ea
- #EPP1931 16"W x 16"D77.75 ea

w/T-Gel Overlay

- #EPP1974 16"W x 16"D\$138.00 ea
- #EPP1975 18"W x 16"D138.00 ea

More Anti-Thrust Cushions p29

Never lose a

Stow wheelchairs out of the way at night,



Stay-Put™ Cushion

The Stay-Put Cushion is a traditional wheelchair seat cushion that is "hinged" in the center and is securely fastened under the sling seat to prevent the cushion from being lost or improperly positioned on the wheelchair. When the wheelchair is folded (for storage or transport), the cushion stays attached to the wheelchair.

Who needs a Stay-Put™ Cushion?

Stay-Put cushions are ideal for facilities that are interested in saving time and money. Cushions can be lost or misplaced when wheelchairs are stowed out of the way at nighttime. Besides cost, there can be infection control and patient care issues if cushions are placed on the wrong resident's chair or placed incorrectly on the chair. Caregivers never have to worry about losing or misplacing a cushion again—saving the facility valuable time and money. When the patient needs the wheelchair, the caregiver simply opens it, and the cushion is already properly attached for use.



Stay-Put cushions fit most sling-seat wheelchairs and come in a variety of options to suit your specific patients' needs.



 For more information, and to see a video, go to <http://Video.AliMed.info> and click on Stay-Put Cushion.



Open for use, cushion stays in place



Built-in hinge allows cushion to fold in the middle

Attaches to wheelchair under sling seat.



Closed for storage, cushion stays attached

What therapists are saying about Stay-Put...

"I am an LPTA who has been working in the PT field for 32 years, ordering all types of rehab equipment for the individuals I serve. I am writing to let you know that the Stay-Put cushion has had such a positive impact on the people that use it, that I feel I have to give credit where it is due. The feedback is that it is comfortable, compact and easy to use, and clean. The less bulk and add-ons that people have to pack up is greatly appreciated. So, right on with the Stay-Put."—Cathy, LPTA

"Stay-Put cushions are ingenious. As a therapist, I can only say, Why didn't I think of that? Such a simple concept, yet such an expert execution of a great idea. These cushions meet the demands of most wheelchair-dependent people, and make transporting and storing wheelchairs so simple. I highly recommend Stay-Put."—Gina, PTA

COMING SOON! Utility Foam

Call 800.225.2610 x103 for details.

cushion again!

cushions are still there in the morning!

Your family of Stay-Put™ Cushions

Pressure relief

T-Foam™ Stay-Put™

This medium temper-foam (memory) cushion is ideal for extended seating because it molds to body shape for the best weight distribution. Its open-cell structure allows air to circulate freely through the material, reducing heat build-up.



#JEE10952 16"W x 16"D x 2"H\$106.75 ea

#JEE10953 18"W x 16"D x 2"H106.75 ea

Pressure redistribution

AeroCell Stay-Put™

The air cushion that's best for pressure redistribution for reducing risk of pressure ulcers. Single Valve AeroCell delivers the same pressure to each cell while Dual Valve enables each side of the cushion. 2"H.



#JEE10960 16"W x 16"D, Single Valve ..\$267.00 ea

#JEE10961 18"W x 16"D, Single Valve267.00 ea

#JEE10962 16"W x 16"D, Dual Valve267.00 ea

Increased sitting tolerance

Sit-Straight™ with Coccyx Stay-Put™

The convex wedge shape fills the hammock sag in a fabric wheelchair seat. It helps reduce forward sliding and provides a flatter and more supportive positioning surface. Cutout reduces pressure on the coccyx, which increases sitting tolerance.



#JEE10954 16"W x 16"D x 3"H\$123.75 ea

#JEE10955 18"W x 16"D x 3"H123.75 ea

Shear reduction

T-Gel™ Stay-Put™

Consists of a solid top sheet of shear-reducing polymer gel, similar in consistency to human flesh, over an underlay of temper (memory) foam and a solid seat base. The gel reduces shear while the temper foam optimizes weight distribution on the flat seating surface. Ideal for reducing shear and pressure.



#JEE10956 16"W x 16"D x 3"H.....\$205.00 ea

#JEE10957 18"W x 16"D x 3"H.....205.00 ea

Stay-Put Cushion for Facilities

Available August 2012

Never lose a wheelchair cushion again!



Stay-Put Cushions

p46



297 High Street, Dedham, MA 02026

ELECTRONIC SERVICE REQUESTED

PRESORTED
STANDARD
U.S. POSTAGE
PAID
AliMed® inc.

©2012 AliMed® inc. All rights reserved. Printed in U.S.A. Specifications, prices, and availability subject to change without notice. \$1.50


Improved posture.
Better skin protection.
Safer sitting.

Introducing our newest Sit-Straight™ cushion!
Now with T-Gel!

 Sit-Straight™ Cushion with T-Gel™


The Problem: Repetitive use of typical flat cushions poses two serious risks for your patients—forward sliding and sacral sitting. A sagging sling seat provides insufficient support, allowing the cushion to further sag in the middle. This causes the patient's body to compensate for an unstable, unsupportive sitting surface, leading to further forward sliding. Sacral sitting contributes to poor sitting posture and leads to a decrease in functional abilities, low sitting tolerance, and increased pressure that can lead to decubitus ulcer formation. It can even cause patients to fall from the wheelchair.

The Solution: Sit-Straight Cushions combat sacral sitting and promote proper positioning by combining two key elements—a **Wedge Cushion** shape and a **Solid Seat Insert** base. The wedge cushion shape keeps the knees positioned slightly higher than the hips, promoting proper posture and effectively limiting forward sliding. The solid seat insert is a firm, convex, tapered foam base that fills the sag created by the wheelchair sling seat, providing a level, stable sitting surface without raising the overall height of the cushion surface.

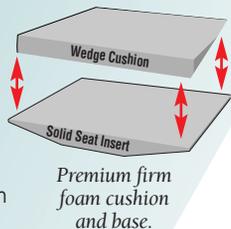

For additional comfort and skin protection, we have added a layer of our checkerboard T-Gel elastomer gel. T-Gel is a soft, solid, rubber-like polymer that mimics fatty tissue, minimizing the destructive effects of friction and skin shear.

- Skin protection**
- Positioning**
- Fall prevention**

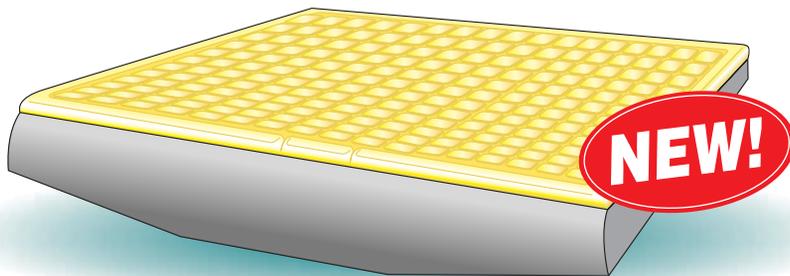

Sit-Straight™ with T-Gel Cushion Two of AliMed's most popular cushions have come together to form our newest skin protection and positioning cushion. The new Sit-Straight with T-Gel Cushion combines the integrated wedge and solid seat design of our best-selling Sit-Straight Cushion with the unbeatable shear-reducing qualities of our checkerboard T-Gel topper. The result is a multi-purpose cushion offering comfort, postural support, and skin protection with excellent air circulation. Add a stretch-knit, water-resistant cover with a nonskid base and rear safety ties and this cushion can't be beat. All this for under \$200.00!

#JEE11009 18" x 16" ...\$167.75 ea.....\$161.75 ea (6+)
#JEE11010 16" x 16"170.75 ea.....164.75 ea (6+)



ORDER NOW! CLICK www.AliMed.com CALL 800-225-2610 FAX 800-437-2966