



# Instructions for Measuring Perfect Fit™ Aprons


- **Chest (C):** Measure circumference around widest part of chest.
- **Waist (W):** One-Piece: measure waist. Two-Piece: measure where you want the kilt top (this may or may not be your natural waist).
- **Hip (H):** Measure circumference around widest part of hips.
- **Hem:** We suggest 1" above the knee

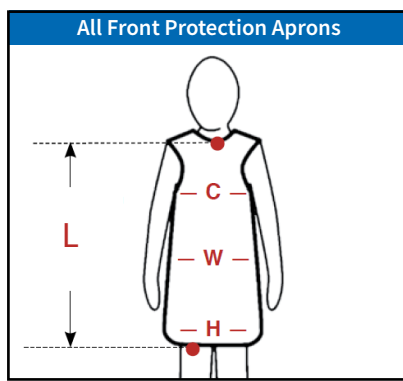

**Front Protection Apron** length is measured from sternal notch to hem (We recommend 1" above the knee).

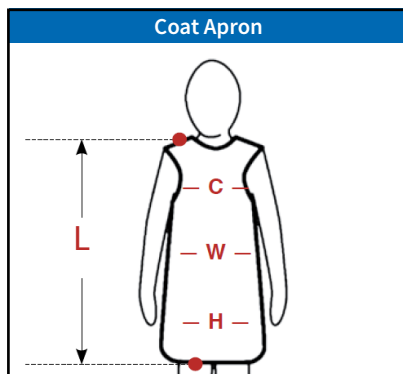

**Coat Apron** length is measured from top of shoulder to hem.

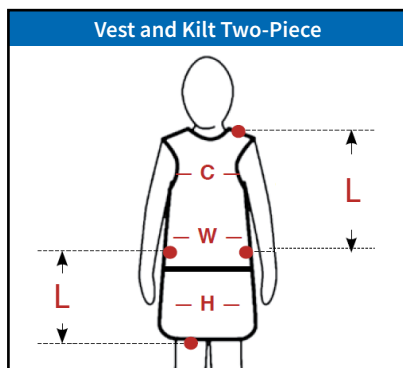

**Vest and Kilts** For vest, measure from top of shoulder to top of kilt. Add 3" to vest for overlap of vest/kilt. For kilt, measure from top of kilt to hem.

Visit [AliMed.com/radiation](https://www.alimed.com/radiation) for more about Perfect Fit Aprons.