




Motion Sensor Night Light, #712271

Instructions for Use and Care

Description


This system automatically lights a path for patients who get up in the night. It may be used at the bedside, in the bathroom, or wherever additional light is needed. The Motion Sensor Night Light (*Figure 1*) can be used by itself or with the rest of the lights included in the package. The Night Light is activated as soon as the passive infrared (PIR) sensor detects the motion of a patient's body. Once motion is detected, the unit's LED lights illuminate the floor.

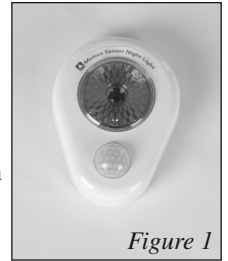

Contents

This package contains (*Figure 2*):

- Motion Sensor Night Light
- Base Unit
- Adjustable Bracket
- Batteries (not shown)

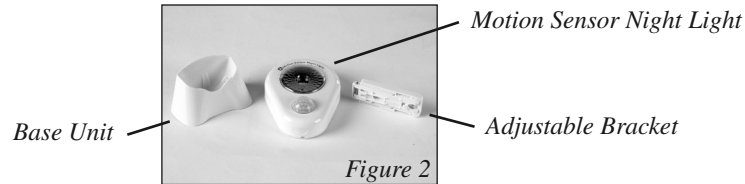

Setup

Installing the Batteries:

1. Remove rear cover by pressing clip at the top (back) and pulling forward (*Figure 3*).
2. Install three alkaline AA batteries in slots provided.
3. Replace rear cover so that it snaps into place.



Mounting

Installed position should have a minimum distance of 18" from the floor and 2½" from a corner. Install vertically on wall.

Base Mounting:

Place Motion Sensor Alarm into Base Unit (*Figure 4*) and place on a stable surface so that lights can be seen on the floor.

Wall Mounting:

1. Attach Adjustable Bracket to the back of the motion sensor alarm (*Figure 5*).
2. Fasten the adjuster to the wall with two-sided tape or screws (not included).
3. If using double-sided tape, ensure that wall has been cleaned of dirt, oil, or water.



Testing

To ensure proper function, perform test after installation and after the batteries have been replaced.

1. Turn to Auto Mode. Pass hand in front of sensor detector. Motion Sensor Night Light will illuminate and continue to illuminate for a minimum of 7 seconds or as long as PIR Sensor is activated.
2. If sensor does not function properly, check batteries. If batteries are OK but unit continues to function improperly, discontinue use.

Technical Specifications

Power Source: Three AA Batteries

Sensor Angle: 140°

Switch State: Auto/ON/OFF

Detection Sensor: PIR Sensor

Working Temperature: -4°F to 113°F

Brightness Sensor: Photoelectric Sensor

Alarm Time: 7-10 seconds

Size: 6 ½"W x 4 ¾"L x 6 ½"H

Detection Distance: 9'

Care

Clean Motion Sensor Night Light with mild detergent and/or disinfectant. Use clean cloth or sponge. Do not launder or submerge in any liquid.

Warning

Before using this device, read the instructions in their entirety to familiarize yourself with the product. Proper installation of the Motion Sensor Night Light will help ensure optimal operation.

- Examine the device for potential damage or wear prior to use. Do not use if damage is visible or parts are missing.
- Do not modify product.
- Do not stare at bulb.
- Keep from high-temperature sources (for example, fire, heating devices, and air-conditioning engines) and open flame.

PIR Sensor Cautions

The sensor's sensing ability could be lowered if the following conditions are present:

- The temperature of the atmosphere is too similar to the patient.
- Body heat cannot be detected due to excessive clothing.
- Patient walks directly toward the sensor.
- Mount is in an unstable location.

Construction

ABS Plastic

To Reorder

712271 Motion Sensor Night Light, 3/pk

Also Available

Sensor Pad Systems alert caregivers to a potential fall by sounding an audible alarm when a patient attempts to get out of bed, attempts to stand, or slides out of a seat. Alarm is triggered when patient's body weight is removed from the pad. Use with confused, nonambulatory patients, or sedentary patients with mild to moderate balance deficits.

Our rugged, heavy-duty fall alarms feature durable plastic cases to increase longevity and reliability. The **TR2**, our most tamper-resistant unit, incorporates added features such as hidden dual reset buttons to limit patient tampering.



Replacement Alarm Units

#AXX71 0985 TR2 Alarm
#AXX71 1290 IQ Duo Plus Alarm
#AXX2032 VOICE Alarm

Replacement Pads

#AXX79366 Bed Pad
#AXX79367 Bed Pad, 10/pk
#AXX77061 Chair Pad (6-month)
#AXX77063 Chair Pad (6-month), 10/pk
#AXX75991 Chair Pad (18-month)
#AXX75992 Chair Pad (18-month), 10/pk

Manufactured for:

 **AliMed® inc.**
Dedham, MA 02026-9135

www.AliMed.com

#6725REV0811 CCO 409
©2011 AliMed® inc. All rights reserved.
GTIN:733657200805
Made in China