

Worry-Free® Sensor Alarm #71 3307

Instructions for Application, Use, and Care

Your new AliMed® Worry-Free Sensor Alarm features a sealed, tamper-free battery compartment, and includes a two-year battery life guarantee. It also includes a belt clip and an adjustable strap that can be used to attach the alarm to a chair or a bed.

Initial Use:

When unpacking the Worry-Free™ Sensor Alarm for the first time, you will notice a small plastic tab protruding from the side of the alarm. This tab prevents the battery from draining prior to purchase. Before using the alarm, please pull this tab, then remove and discard. This activates the battery; the alarm is now on and ready to use.

Operating Instructions:

- Connect the Worry-Free Sensor Alarm to a bed/chair sensor pad, seatbelt, roll belt, or sensor cushion by simply plugging the connector into the jack near the bottom of the alarm.
- The system is now ready to use.
- Choose a high or low volume tone by placing the switch on the side of the alarm in the corresponding HI or LO position.

Test Alarm:

- For sensor pads: place firm pressure on the pad for at least 3 seconds. The alarm will emit two short beeps. When pressure is removed, the alarm will sound.
- For seatbelts and other similar devices: If seatbelt was already closed, open it. The alarm will sound. If seatbelt is open, close the seatbelt and you will hear the alarm emit two short beeps. When the seatbelt is opened, the alarm will sound.
- Note: If patient is placed on the sensor pad or buckled into a seatbelt and attempts to remove the cord from the alarm, the alarm will sound.
- To reset the alarm, place pressure back on pad, close the seatbelt or similar device, or simply press the RESET button on the side of the alarm.
- Low-battery red LED will light, along with a short beep to indicate a low battery. When this occurs, the alarm may be discarded. However, the alarm will continue to function properly under low battery for at least 5 days, allowing you time to replace it.
- **Test alarm with sensor device before each use to verify that both alarm and sensor are operating properly to ensure patient safety.**
- Discontinue use immediately if unit is damaged or no longer functions.
- **NOTICE: OPENING THE ALARM OR TAMPERING WITH BATTERIES WILL VOID THE WARRANTY.**

Care:

- Wipe clean using standard cleaning disinfectants.
- Do not immerse in fluids.
- Check function before each use. Do not use equipment that has been damaged.



Manufactured for:



AliMed® inc.
297 High St., Dedham, MA 02026 USA
PH 800.225.2610 • FAX 800.437.2966



MDSS GmbH
Schiffgraben 41
30175 Hannover,
Germany

AliMed.com

#7140REV0416 CCO832

©2016 AliMed® inc. All rights reserved.

Made in China