

AliMed®

IQ Duo Plus Alarm, #711290

Instructions for Use and Setup

Use

Your new AliMed IQ Duo Plus Alarm is supplied with three AAA batteries and will be ready for use when you install them. This alarm control unit supports sensor devices, including bed and chair sensor pads, seatbelt alarms, wheelchair alarms, and seat cushions. It also features a magnetic pull-cord.

Setup

Battery Installation:

- Open battery compartment by loosening the screw on the base of the cover.
- To install, insert battery with flat base on negative contact spring. Press battery into slot compressing spring.


Sensor Function:

- Open battery compartment by loosening the screw on the base of the cover.
- Slide function switch to **Sensor**.
- Insert sensor cable into slot located at the top of the alarm. **Test alarm.*** The alarm is now ON and ready for use.

Magnet Function:

- Open battery compartment by loosening the screw on the base of the cover.
- Slide function switch to **Magnet**.
- Attach alligator clip to patient's clothing.
- Adjust cord length. **Test alarm.*** The alarm is now ON and ready for use.

REMINDER: Before using, remove the "reset button" sticker on face of alarm.

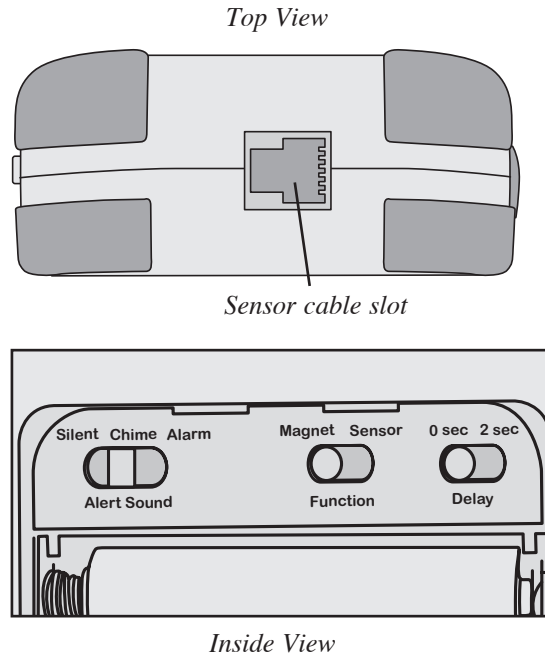


Testing

*Test alarm daily to ensure patient safety.

To test alarm as a magnetic alarm, set function switch behind battery cover to "Magnet." Attach magnet to front of alarm. Remove magnet. If alarm sounds, it is ready for use.

To test alarm with a sensor device such as a sensor pad, set function switch to "Sensor." Insert sensor device cable into sensor cable slot. Apply pressure to sensor pad by pressing on pad with hand, remove. If alarm sounds, it is ready for use.

NOTE: To reset alarm, press the AliMed logo located on the front center of the alarm.

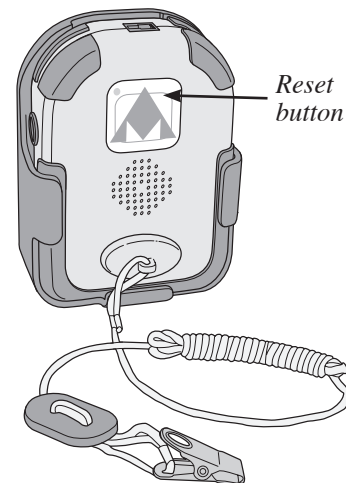

Manufactured for:

AliMed® Inc.
297 High St., Dedham, MA 02026 USA
PH 800.225.2610 • FAX 800.437.2966

AliMed.com
#6158REV0617 CCO885
©2017 AliMed® Inc. All rights reserved.
Made in China

Customized Setup Options

In the battery compartment, there are different setup options available.

Function: Magnet or Sensor (see reverse for instructions).


Delay: 0 or 2 seconds—set to **0 sec** for immediate detection of movement. Set to **2 sec** to allow patients time to readjust themselves.

Volume: 85-dB or 100-dB switch on side of alarm—set to **85 dB** for a quieter setting. Set to **100 dB** for a louder alarm if there are other noise-generating devices present. A silent mode option is also available.

Alert: **Silent**, **Chime**, and **Alarm**. Silent setting can be used with the nurse-call jack connection. When used this way, the alarm sounds an alert only at the nurse-call station.

NOTE: Silent mode should only be used in conjunction with a nurse-call system (connected to the IQ Duo Plus). **Do not** use the IQ Duo Plus in silent mode for stand-alone operation, as no alarm tone will sound. If used in stand-alone operation, silent mode may also lead to premature battery depletion. **When using the alarm unit in silent mode, be sure to reset the alarm each time it is triggered. Resetting only the nurse-call system will dramatically shorten the alarm's battery life.**

For video instructions, visit AliMed.com/alimed-videos.
For alarm accessories, visit AliMed.com.

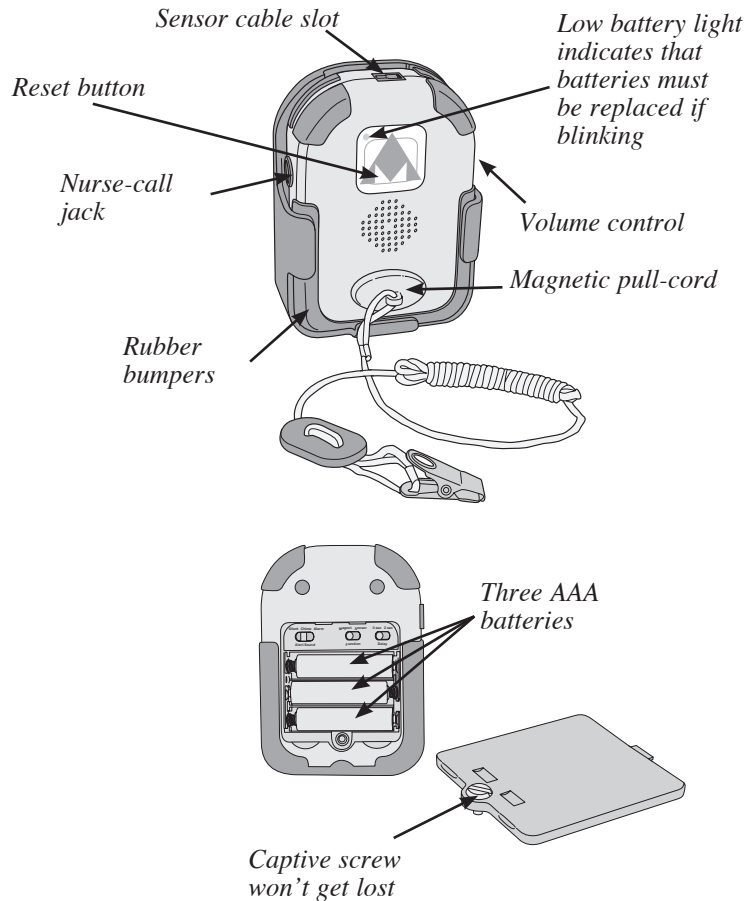

To Reorder

#711290	IQ Duo Plus Alarm
#79366	Bed Sensor Pad, 6-Month Warranty
#79367	Bed Sensor Pad, 6-Month Warranty, 10/cs
#77061	Chair Sensor Pad, 6-Month Warranty
#77063	Chair Sensor Pad, 6-Month Warranty, 10/cs
#711533	Battery Door Cover, 10/pk
#74878	AAA Batteries, 24/pk



Manufactured for:



AliMed® Inc.

297 High St., Dedham, MA 02026 USA
PH 800.225.2610 • FAX 800.437.2966



MDSS GmbH
Schiffgraben 41
30175 Hannover,
Germany

AliMed.com
#6158REV0617 CCO885
©2017 AliMed® Inc. All rights reserved.
Made in China